



## The Welch Allyn OAE Hearing Screener



Objective hearing screening that  
gives you an accurate response,  
so your patient doesn't have to.

**WelchAllyn®**

Advancing Frontline Care™



## Hearing screening that's accurate, objective, and easy to use

Detect hearing loss, prevent language disabilities, and improve patient outcomes with the Welch Allyn OAE Hearing Screener, the child-friendly hearing screener that's objective, accurate, and easy to use. By requiring minimal patient cooperation, the OAE Hearing Screener lets you precisely and objectively screen for hearing loss in newborns, infants, toddlers, preschool, and school-age children in just seconds. And with simple, intuitive operation and clear, easy-to-understand "pass/refer" reporting in a portable design, the OAE Hearing Screener effectively addresses many of the difficulties associated with conventional hearing screening.



Newborns



Infants



Toddlers



Preschoolers

## Detect hearing loss before significant speech impairments occur

Significant hearing loss can develop after the newborn period, so birth screening is not enough. Regular hearing screening is important throughout childhood. The American Academy of Pediatrics recommends that all infants and children, in addition to universal newborn hearing screening, should periodically receive objective screenings for hearing impairment through age 10. Designed to be safe and comfortable for use on children of all ages, the OAE Hearing Screener makes hearing screening simple, safe and accurate.



## Medical Device Depot Inc.

www.medicaldevicedepot.com  
Toll Free 1-877-646-3300

## Simple Testing. Clear Results.

After administering the hearing exam, the OAE Hearing Screener shows the caregiver a result of "pass" or "refer" on a large, easy-to-read screen. Data can be stored for later viewing with the OAE Hearing Screener's 10-test memory, or printed immediately for the patient's record.

### Label Printer

The OAE Hearing Screener comes equipped with a label printer to streamline patient data documentation. Print results from both ears on a single self-adhesive label, and place it on the patient's record chart to immediately document patient information.

5000 HZ DP 14  
PASS NOISE -15

5000 HZ DP -20  
REFER NOISE -20



### OAE Disposable Ear Tips

The OAE Hearing Screener can be fitted with a variety of ear tip sizes, further ensuring the accuracy of device and your readings. Tips are sized to fit newborns, infants, toddlers, preschool and school-age children.

## With the Welch Allyn OAE Hearing Screener, you get:

- > Fast, accurate results—about 10 seconds per ear
- > Objective testing—no patient response required
- > Clear results that don't require caregiver interpretation—Pass/Refer result provided by instrument
- > Portable, reliable design gives you increased confidence and flexibility
- > 10-test memory storage in device
- > Simple printer interface for easy patient data documentation
- > Patient information can be downloaded into a PC ASCII file
- > Optional power supply: rechargeable lithium battery or AC power
- > 1-year warranty, extended warranty available

## Reimbursement Information

The OAE Hearing Screener is eligible for Medicare Reimbursement using CPT code 92587 evoked otoacoustic emission, limited.

### A WORD ABOUT THE CODING INFORMATION CONTAINED IN THIS BROCHURE FROM THE CURRENT PROCEDURAL TERMINOLOGY, FOURTH EDITION (CPT®).

Please note that the coding information contained here is neither conclusive nor does it ensure that you will be successful in obtaining coverage and/or reimbursement from government and private payers.

Depending on circumstances that will vary from patient encounter to patient encounter, separate reimbursement may or may not be allowed for use of a Welch Allyn product used during a specific procedure or service. Use of the product may be included in the reimbursement for Evaluation and Management services provided by the physician. Please check with the specific payer to confirm whether or not the use of a separate code is acceptable. Welch Allyn does not assume any responsibility for the decisions made by payers regarding payment for procedures involving our devices when the coding information suggested in this brochure is used.

Current Procedural Terminology (CPT) is copyright 2003 American Medical Association. All rights reserved. No fee schedules, basic units, relative values, or related listings are included in CPT. The AMA assumes no liability for the data contained herein. Applicable FARS/DFARS restrictions apply to government use.



School-Age Hearing Screening

## Ordering Information

### OAE HEARING SCREENER SET

29400 Welch Allyn OAE Hearing Screener Set

### PRINTER

29450 OAE Label Printer

29451 OAE Label Roll Replacement (Self-adhesive) 220/roll

### OAE ACCESSORIES

29440 OAE Carrying Case (Optional case for Screener Set Only, No Printer)

29410 OAE Test Cavity, 5/bag

29411 Ear Probe Body Nozzle, OAE, 5/bag

29460 Extended Warranty—One Year (covers hearing screener, unit, probe, calibration once every two years, battery replacement)

### OAE DISPOSABLE EAR TIPS

29420 OAE Ear Tip Sampler Pack (Newborn, Infant, Pediatric, Adolescent)

29421 OAE Tree Ear Tips, Newborn, 20/bag

29422 OAE Tree Ear Tips, Newborn, 100/bag

29423 OAE Blue Infant Ear Tips, 20/bag

29424 OAE Blue Infant Ear Tips, 100/bag

29425 OAE Pediatric Foam Ear Tips, 20/bag

29426 OAE Pediatric Foam Ear Tips, 100/bag

29427 OAE Adolescent Foam Ear Tips, 20/bag

29428 OAE Adolescent Foam Ear Tips, 100/bag



29440 OAE Hearing Screener Carrying Case (optional)



29420 OAE Ear Tip Sampler Pack

For more information on the OAE Hearing Screener or an in-office demonstration, contact your local distributor or call Welch Allyn at 1.800.535.6663.

**WelchAllyn®**

Advancing Frontline Care™

**Medical Device Depot Inc.**  
www.medicaldevicedepot.com  
Toll Free 1-877-646-3300