Video Graphic Printer

取扱説明書	Page 2	J
Instructions for Use	Page 54	EN
Mode d'emploi	Page 104	F
Gebrauchsanweisung	Seite 154	D

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Owner's Record

The model and serial numbers are located at the rear. Record these number in the space provided below. Refer to these numbers whenever you call upon your Sony dealer regarding this product.

Model No. _____
 Serial No. _____

WARNING

To prevent fire or shock hazard, do not expose the unit to rain or moisture.

To avoid electrical shock, do not open the cabinet. Refer servicing to qualified personnel only.

Symbol on the products


This symbol indicates the equipotential terminal which brings the various parts of a system to the same potential.

Important safeguards/notices for use in the medical environments

1. All the equipments connected to this unit shall be certified according to Standard IEC601-1, IEC950, IEC65 or other IEC/ISO Standards applicable to the equipments.
2. When this unit is used together with other equipment in the patient area*, the equipment shall be either powered by an isolation transformer or connected via an additional protective earth terminal to system ground unless it is certified according to Standard IEC601-1.

* Patient Area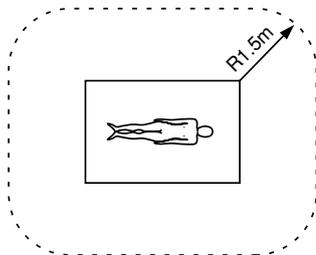

3. The leakage current could increase when connected to other equipment.
4. This equipment generates, uses, and can radiate frequency energy. If it is not installed and used in accordance with the instruction manual, it may cause interference to other equipment. If this unit causes interference (which can be determined by unplugging the power cord from the unit), try these measures: Relocate the unit with respect to the susceptible equipment. Plug this unit and the susceptible equipment into different branch circuit. Consult your dealer.

For the customers in the U.S.A.

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

You are cautioned that any changes or modifications not expressly approved in this manual could void your authority to operate this equipment.

This device requires shielded interface cables to comply with FCC emission limits.

For the customers in Canada

This unit has been certified according to Standard CSA C22.2 No.601.1.

Warning on power connection for medical use

Please use the following power supply cord. With connectors (plug or female) and cord types other than those indicated in this table, use the power supply cord that is approved for use in your area.

	United States	Canada
Plug Type	HOSPITAL GRADE*	HOSPITAL GRADE*
Female end	E62405, E35708	LR53182, LL022442, LL088408
Cord type	E159216, E35496 Min.Type SJT Min.18AWG	LL112007-1, LL20262, LL32121, LL84494 Min.Type SJT Min.18AWG
Minimum cord set rating	10A/125V	10A/125V
Safety approval	UL Listed	CSA

*Note: Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked 'Hospital Only' or 'Hospital Grade'.

Table of Contents

Introduction	About This Manual	56
	Overview	57

Operation	Loading Paper	58
	Making Printouts	61
	Printing a Picture on One Page - Single Picture Mode ..	61
	Printing Multiple Images on One Page - Multi Picture Mode	64
	Making Printout Variations	70
	Cutting the Paper Manually	71

Connection and Adjustment	Connection	73
	Setting Up the Printer	74
	Menu Operation	74
	PICTURE Menu	76
	LAYOUT Menu	79
	PRINTER Menu	83
	Adjusting the Printout Size	88
	Saving the Menu Settings	92

Others	Precautions	94
	Maintenance	95
	Paper	97
	Specifications	98
	Troubleshooting	99
	Error Messages	100
	Location and Function of Parts and Controls	101
	Front	101
Sub Panel	102	
Back	103	

About This Manual

This manual is divided into four chapters. This section explains the organization of this manual.

Introduction

Describes the features of the monochrome video graphic printer.

Operation

Describes actual printing **once all connections and adjustments have been made**, as explained in the next chapter. You will be able to make printout variations after reading through this chapter.

Connection and adjustment

Describes how to make connections and make adjustments using the menus displayed in the printer window display. Once all connections and adjustments have been made, there should be no need to perform these operations again during normal printing operations. These operations must, however, be performed after reinstalling, or if the picture quality degrades, or if adjustment becomes necessary.

Others

Notes the precautions to be observed when using the printer, lists errors and their handling, and explains troubleshooting. Also provided is information on the locations and functions of parts and controls.

Conventions used

Cross reference

Throughout this manual you will find the references to other sections of the manual that contains related information.

Important note

Be sure to read the sections of the manual marked **Note**. They explain points that you should be aware of to operate the printer correctly and prevent malfunctions.

The UP-980/980CE is a monochrome video graphic printer that reproduces images from video equipment. Large size pictures can be printed out quickly and easily using the following features:

Clear and stable print quality

- 256 gradations of black and white.
- Stable printouts using the temperature compensation technology

Various printout modes


Single picture mode

Side mode

Reverse mode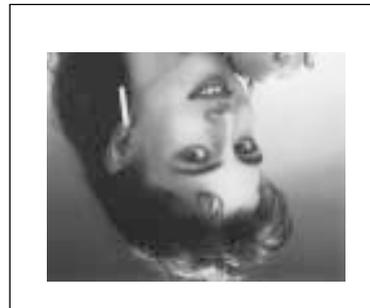

Multi-picture mode

In addition to the above variations, you can make variety of printouts by changing menu settings of the printer.

Multiscan capability

- You can use a color composite video signal (NTSC/PAL compatible), a black and white video signal (EIA/CCIR compatible), or a 31.5 kHz high scan signal (horizontal scanning frequency) as the input signal.
- In the auto scan mode, the printer automatically sets the print area according to the type of the input signal.

Saving/loading printer specifications

You can put up to three kinds of printer settings made on menus into memory and load any one of the three. The printer operates based on the loaded settings.

Auto-cut function

The paper is automatically cut when printing is finished.

Various types of paper


- The UPP-210HD High Density Printing Paper produces a high density printouts.
- The UPT-210BL Blue Thermal Transparent Film allows you to print images on transparent film.

Loading Paper

Notes

- Do not fold the paper or touch the printing surface. Any dust on the printing surface will result in poor printing quality.
- Use only paper made specially for the UP-980 series (page 97).
- Select the paper type from the PICTURE menu (page 76).


- 1** Press the power ON/OFF switch to turn on the printer.

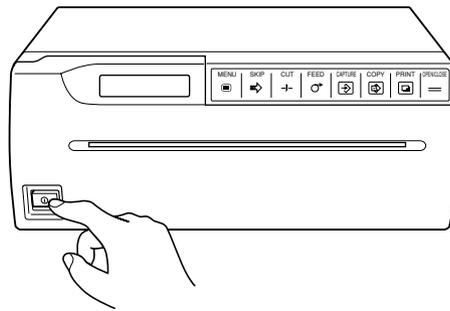The following message appears on the printer window display.


- 2** Press the OPEN/CLOSE button to open the door panel.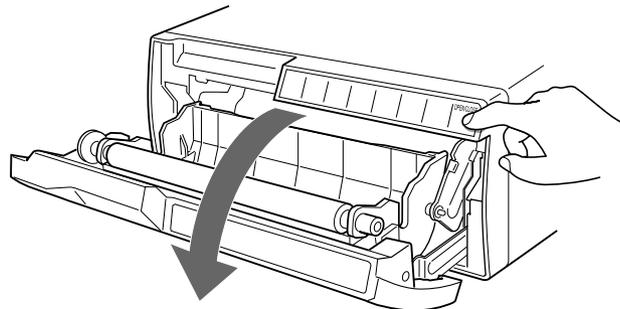

3 Place the paper roll in the printer.


When the UPP-210SE/210HD is used:

Place the paper with the thermo-sensitive side (printing side) facing up.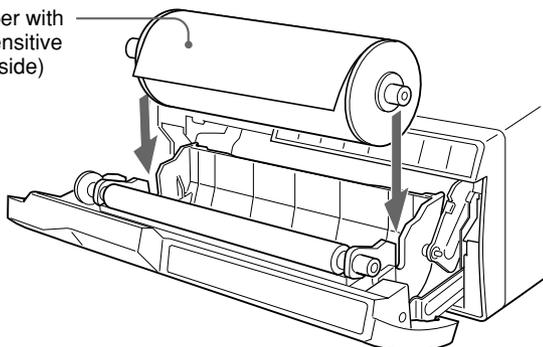

When the UPT-210BL is used:


Place the paper with the thermo-sensitive side (printing side) facing up.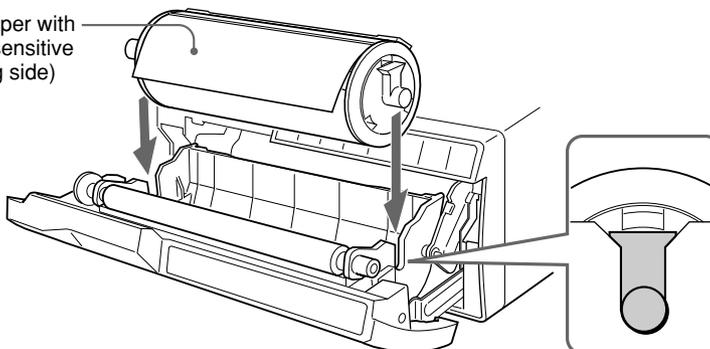

4 Insert the end of the paper into the paper insertion slot manually and feed the paper until its end comes out from the paper outlet.


Note

Be sure to set the paper in the middle of the paper insertion slot and between the guides to avoid any paper jamming.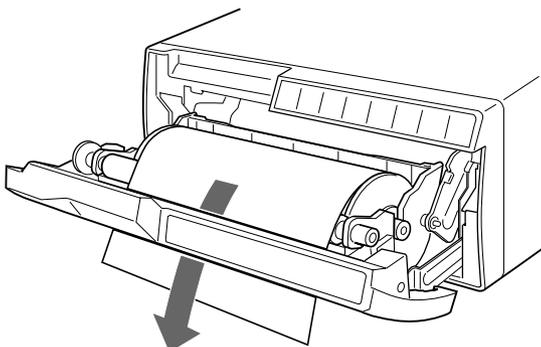

Note

Be sure to remove any slack in the roll when pulling out the paper or film.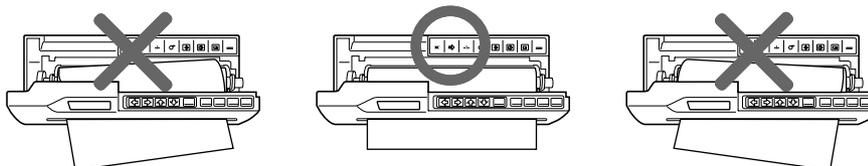


Continue to next page →

Loading Paper (Continued)

- 5 Press the OPEN/CLOSE button to close the door panel.
You can also close the paper lid by simply pushing it.

Notes

- After loading the paper roll or film, press and hold down the FEED button until 15 to 20 cm (6 to 7 $\frac{7}{8}$ inches) of the paper protrudes, then press the CUT button to cut the paper.
- When using the UPT-210BL after having used the UPP-210SE/HD, be sure to clean the thermal head. Otherwise, horizontal stripes may appear on the printouts.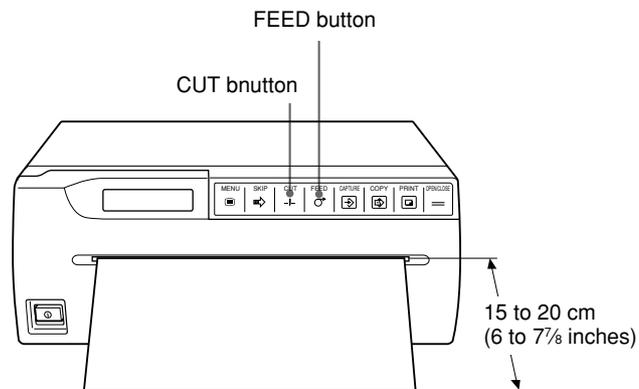

Making Printouts

This section explains the basic printing operation in the auto scan mode. In this mode, the printer automatically sets the print area according to the type of the input signal. As a result, the best possible printouts can be obtained simply by pressing the PRINT button or COPY button.


The printer provides the following two print modes.

- Single picture mode
A full-size image is printed on one page.
- Multi picture mode
Multiple reduced-images are printed on one page.

Printing a Picture on One Page - Single Picture Mode

Before making a full-size printout

- Are the connections correct? (page 73)
- Is the auto scan mode set to ON? (page 87)
- Is the auto cut mode set to ON? (page 86)
You can set the desired amount of the paper to be fed in the auto feed mode (page 80).
- Is the paper roll or film loaded properly? (page 58)
- Is the paper type set correctly? (page 76)
- Is the printer set to import one image into its memory (single picture mode)? (page 81)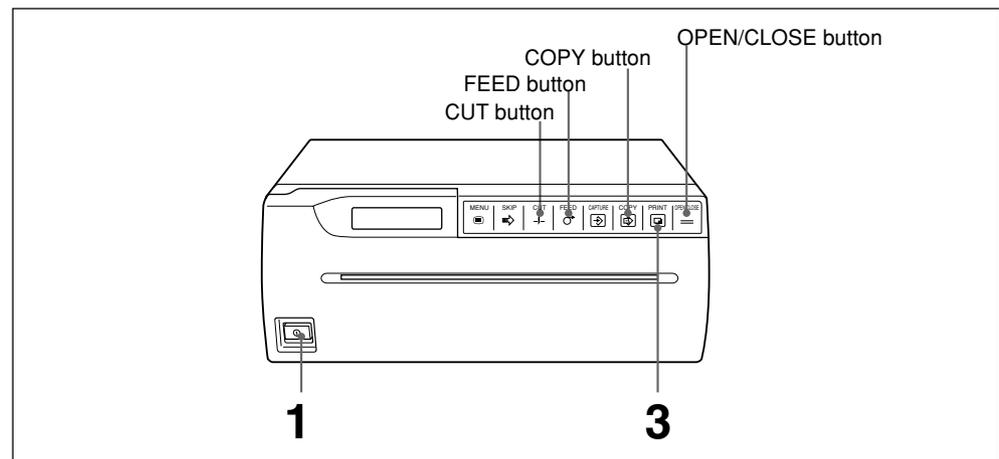


- 1** Press the power ON/OFF switch to turn on the printer.
All buttons except CAPTURE button on the front panel will light.
- 2** Start the video source.
This is done using the controls on the source video equipment.

Continue to next page →

Making Printouts (Continued)

- 3 When the image you want to print appears on the video monitor, press the PRINT button.

The image displayed at the instant you press the PRINT button is captured into memory and is printed out immediately.

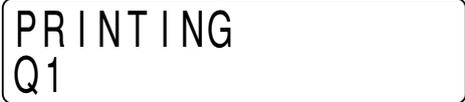PRINTING
Q1

Notes

- In the single picture mode, the CAPTURE button is deactivated.
- In the auto cut mode, remove the paper immediately after printing is completed. If the cut paper covers the paper outlet, jamming may occur.

To make multiple copies of identical printouts

You can make up to 10 identical printouts.

Set the desired number of copies on the PRINT QTY sub menu of the PRINTER main menu. (See page 83.)

The printer makes the chosen number of identical printouts when the PRINT button is pressed.

To copy the last printout again

The image of the last printout is retained in the printer's memory until you press the PRINT button again or turn the power off.

You can make printouts of this image as many times as you like by pressing the COPY button.

If you require multiple copies, set the desired number on the PRINT QTY sub menu and press the COPY button.

To interrupt printing

Press one of the FEED, the CUT or the OPEN/CLOSE button while printing or while copying. The printer stops printing.

To stop printing and print another picture displayed on the video monitor

To do this, the INTERRUPT of the INTERRUPT sub menu of the PRINTER main menu must be set to ON. (See page 84.)

Press the PRINT button while printing or copying. The printer stops printing, captures the image displayed at the instant you press the PRINT button, and starts printing the new image.

Note

If you press the COPY button immediately after turning the power on, an alarm buzzer will sound as nothing is stored in the memory.

If the printout image is blurred

A rapidly moving image may be blurred when printed. Should this occur, change MEMORY to FIELD on the MEMORY sub menu of the PICTURE main menu. (See page 77.)

If the brightness and/or contrast of printouts is unsatisfied

You can adjust the brightness and contrast of printouts using the BRIGHTNESS sub menu and the CONTRAST sub menu.

Pressing the BRIGHT button on the sub panel results in accessing the BRIGHTNESS sub menu easily. (See page 78.)

Pressing the CONTRAST button on the sub panel results in accessing the CONTRAST sub menu easily. (See page 78.)

Remotely controlling the printer

You can remotely control the printer using the RM-91 remote control unit or the FS-20 foot switch.

Since the button on the remote control unit or foot switch functions exactly same as the PRINT button, pressing either of them results in an image being captured and immediately printed in single picture mode.

If INTERRUPT is set to ON on the INTERRUPT sub menu of the PRINTER main menu (see page 84), pressing the switch on the remote control unit or foot switch during printing causes the printer to stop, capture the image now displayed and start printing of new image.


Making Printouts (Continued)

Printing Multiple Images on One Page - Multi Picture Mode

In the multi picture mode, 2, 4 or 6 reduced images are printed on one page.

Selecting number of images to be captured in memory

The number of images printed on one page depends on the number of images set on the MULTI PICTURE sub menu.

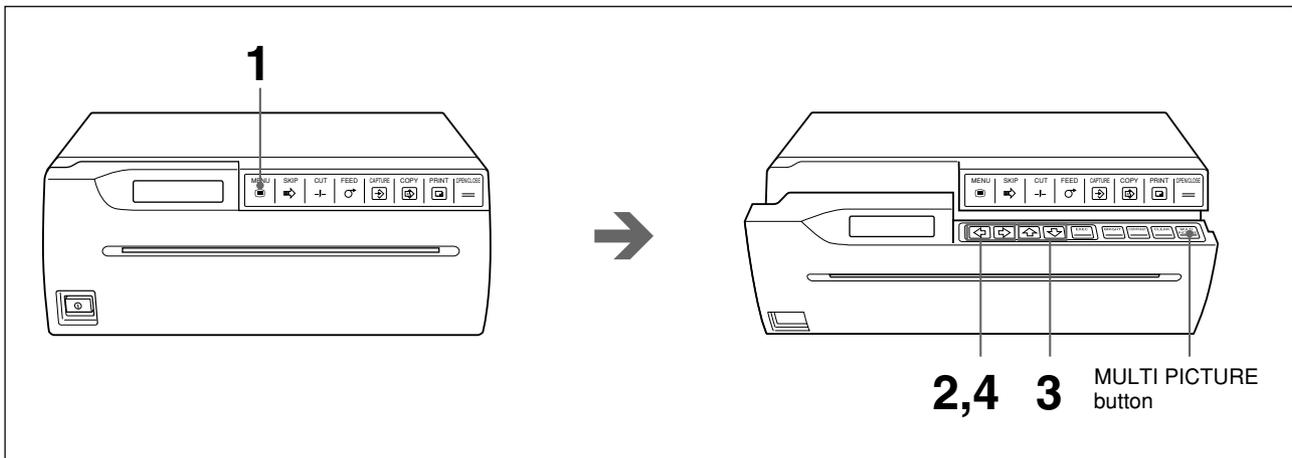- 1** Press the MENU button.
The door panel opens slightly and the sub panel appears.
The main menu appears in the printer window display.

```
MENU
*PICTURE / LAY / PRT
```

- 2** Display the LAYOUT menu by pressing the ⇐ or ⇒ button.

```
MENU
PIC / *LAYOUT / PRT
```

- 3** Display the MULTI PICTURE sub menu by pressing the ⬆ or ⬇ button.

```
MULTIPICTURE
*1 / 2 / 4 / 6
```

-
- 4** Select the desired number of reduced images printed on one page by pressing the ⇐ or ⇒ button.

Displayed number	Number of reduced images
1	1 (full-size)
2	2 (two-reduced images)
4	4 (four-reduced images)
6	6 (six-reduced images)

To return to the regular window display

Press the MENU button to return to the main menu. Then press the MENU button again. The door panel closes and the printer window display returns to the regular display.

Or press the OPEN/CLOSE button. The door panel closes and the printer window display returns to the regular display.

Note

When the printer enters in the multi picture mode, the PRINT button will go off and the CAPTURE button will lights.

To access the MULTI PICTURE sub menu easily

In step 1, the door panel opens slightly and the sub panel appears. Press the MULTI PICTURE button on the sub panel. The MULTI PICTURE sub menu appears.

Image border colors


You can set the border color to either white or black using the FRAME COLOR sub menu. (See page 80.)

Making Printouts (Continued)

Printing multiple images on one page

Before making a printout with multiple images

- Are the connections correct? (page 73)
- Is the auto scan mode set to ON? (page 87)
- Is the auto cut mode set to ON? (page 86)
You can set the desired amount of the paper to be fed in the auto feed mode (page 80).
- Is the paper roll or film loaded properly? (page 58)
- Is the paper type set correctly? (page 76)
- Is the printer set to capture multiple reduced images (multi picture mode)? (page 64)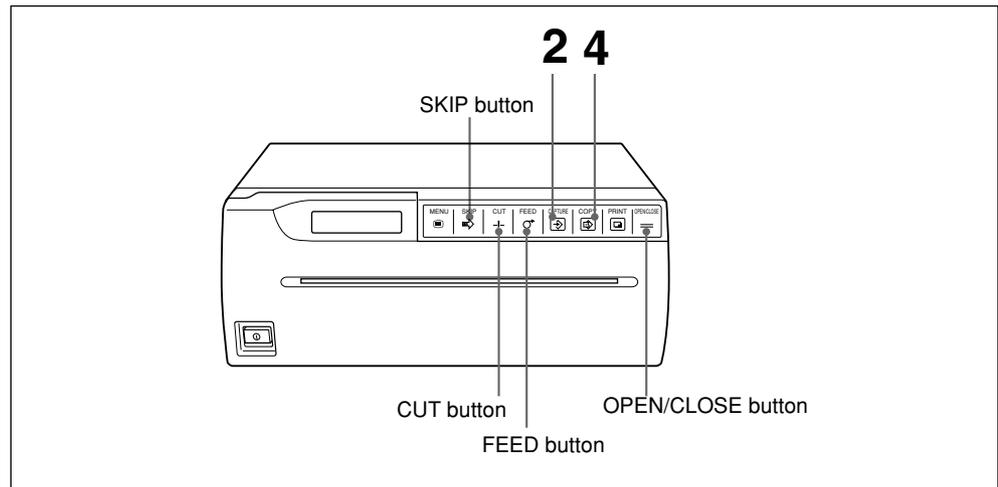


1 Start the video source.

This is done using the controls on the source video equipment. On the printer window display, the following is displayed.

Setting of IMAGE SIZE
(In this case, STANDARD mode is selected)

Underline: Indicates the position at which the cursor is placed. A new image will be captured at this position.

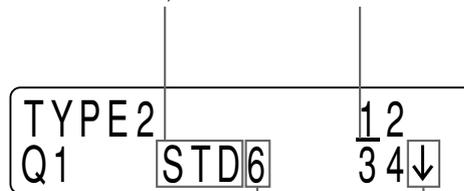Type of multi picture printer mode
(in this case, mode to make a printout with six reduced images)

Indicates that there is the next page.

When SIDE of IMAGE SIZE is selected, the printer window display is as follows.

- 2** When the image you want to print appears on the video monitor, press the CAPTURE button.
The image is captured at the position where the cursor was placed in step 1.
The cursor then moves to the next position.

- 3** Repeat step 2 until you have captured all the desired images.

To replace a captured image

To change the image stored at the third position in an example:

- ① Move the cursor to the third image position by using the SKIP button.

Move the cursor to this position.

- ② Press the CAPTURE button when the new image you want to print appears.
The previously stored image is replaced with the newly captured image.

- 4** Press the COPY button.
All images captured in memory are printed on one page.

Notes

- In the multi picture mode, the PRINT button is not activated.
- In the auto cut mode, remove the paper immediately after printing is completed.
If the cut paper covers the paper outlet, jamming may occur.

To automatically start printing after all images are captured in memory

You can make a printout without pressing the COPY button. To do this, set AUTO PRINT to ON on the AUTO PRINT sub menu of the PRINTER main menu. (See page 87.)

To make multiple copies of identical printouts

You can make up to 10 identical printouts.

Set the desired number of copies on the PRINT QTY sub menu of the PRINTER main menu. (See page 83.)

To copy the last printout again

You can make printouts of these reduced-images as many times as you like by pressing the COPY button.

The printer makes the number (chosen on the PRINT QTY menu) of identical printouts when the COPY button is pressed.

Making Printouts (Continued)

To interrupt printing

Press one of the FEED, the CUT or the OPEN/CLOSE button while printing or while copying. The printer stops printing.

Note

If you press the COPY button immediately after turning the power on, an alarm buzzer will sound as nothing is stored in the memory.

To stop printing and capture another image displayed on the video monitor

To do this, the INTERRUPT of the INTERRUPT sub menu of the PRINTER main menu must be set to ON. (See page 84.)

Press the CAPTURE button while printing or copying. The printer stops printing, captures the new image and positions it where the cursor is placed. Press the COPY button again to make a printout.

If the printout image is blurred

A rapidly moving image may be blurred when printed. Should this occur, change the MEMORY to FIELD on the MEMORY sub menu of the PICTURE main menu. (See page 77.)

If the brightness and/or contrast of printouts is unsatisfied

You can adjust the brightness and contrast of printouts using the BRIGHTNESS sub menu and the CONTRAST sub menu.

Pressing the BRIGHT button on the sub panel results in accessing the BRIGHTNESS sub menu easily. (See page 78.)

Pressing the CONTRAST button on the sub panel results in accessing the CONTRAST sub menu easily. (See page 78.)

Remotely controlling the printer


You can remotely control the printer using the RM-91 remote control unit or the FS-20 foot switch.

Since the button on the remote control unit or foot switch functions exactly same as the CAPTURE button, pressing either of them results in an image being captured. When all the reduced images (the number chosen on the MULTI PICTURE sub menu) have been captured and they are automatically printed in one page.

If AUTO PRINT is set to ON on the AUTO PRINT sub menu of the PRINTER main menu (see page 87), pressing the switch on the remote control unit or foot switch during printing causes the printer to stop, capture the image now displayed and position it where the cursor is placed. If the cursor has been placed at the last position, the printer automatically starts printing.

Clearing all images stored in memory

You can clear all the images in memory at one time.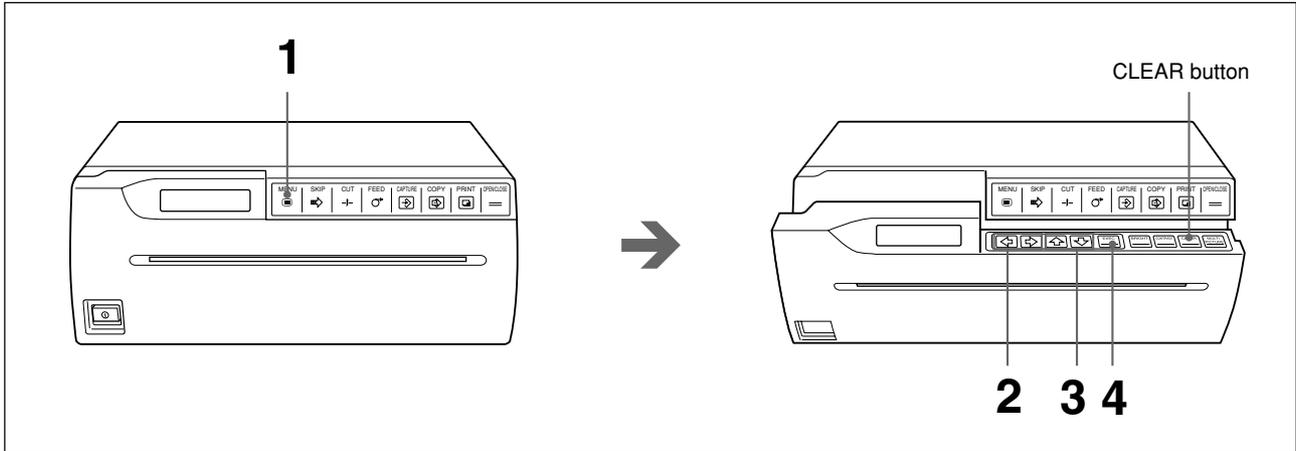

- 1** Press the MENU button.
The door panel opens slightly and the sub panel appears.
The main menu appears in the printer window display.
- 2** Display the PRINTER menu by pressing the \leftarrow or \rightarrow button.

```
MENU  
PIC/LAY/*PRINTER
```

- 3** Display the CLEAR sub menu by pressing the \uparrow or \downarrow button.

```
CLEAR  
          <EXEC>
```

- 4** Press the EXEC button.
All images in memory are cleared.

To return to the regular window display

Press the MENU button to return to the main menu. Then press the MENU button again. The door panel closes and the printer window display returns to the regular display.

Or press the OPEN/CLOSE button. The door panel closes and the printer window display returns to the regular display.


To access the CLEAR sub menu easily

In step 1, the door panel opens slightly and the sub panel appears. Press the CLEAR button on the sub panel. The CLEAR sub menu appears.


Making Printout Variations

In both single picture and multi picture modes, you can make variety of printouts by changing IMAGE SIZE on the IMAGE SEIZE sub menu and setting the number of images to be printed on one page. (pages 79 and 81)

STANDARD mode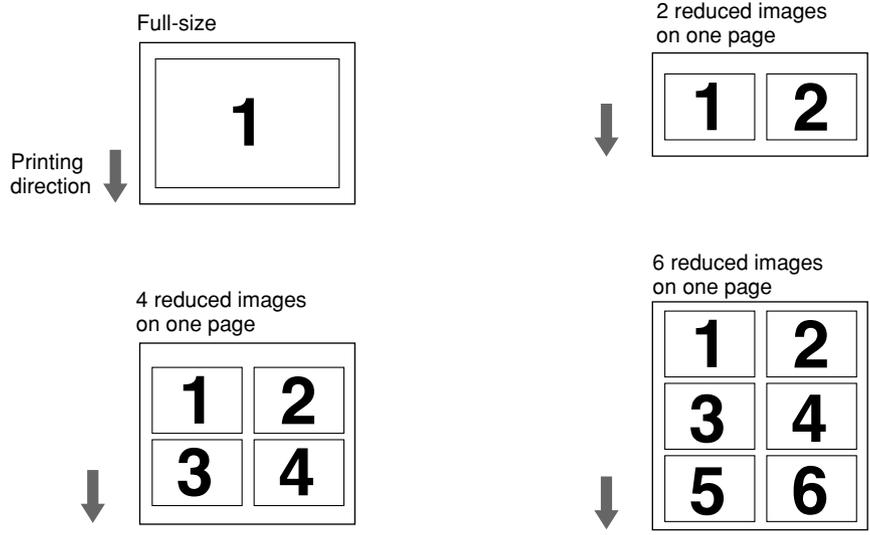


SIDE mode

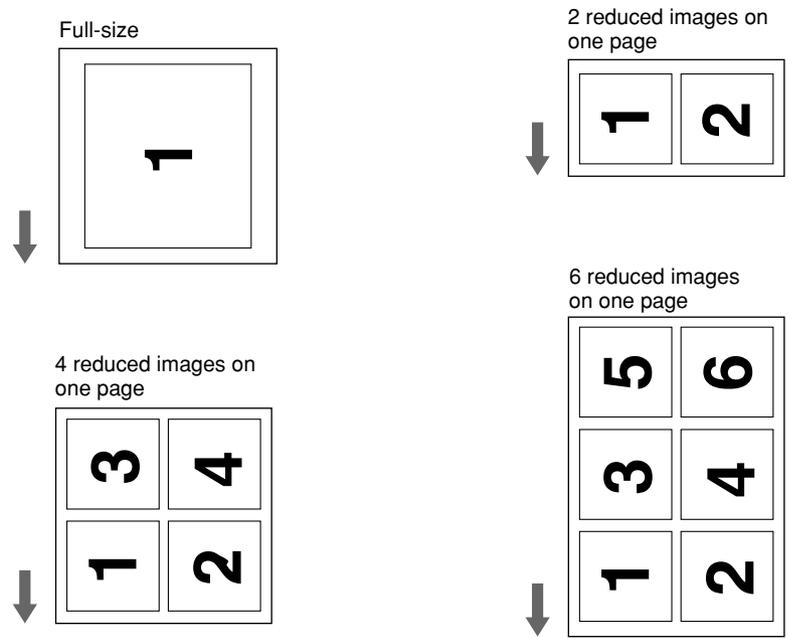In addition to the above variations, you can make printouts upside down (see page 80) and mirror image printouts (see page 81).

Cutting the Paper Manually

In the auto cut mode, the paper is automatically cut each time the printing is completed. However, if you want to print several pictures without cutting the paper, you can cut the paper at any position desired.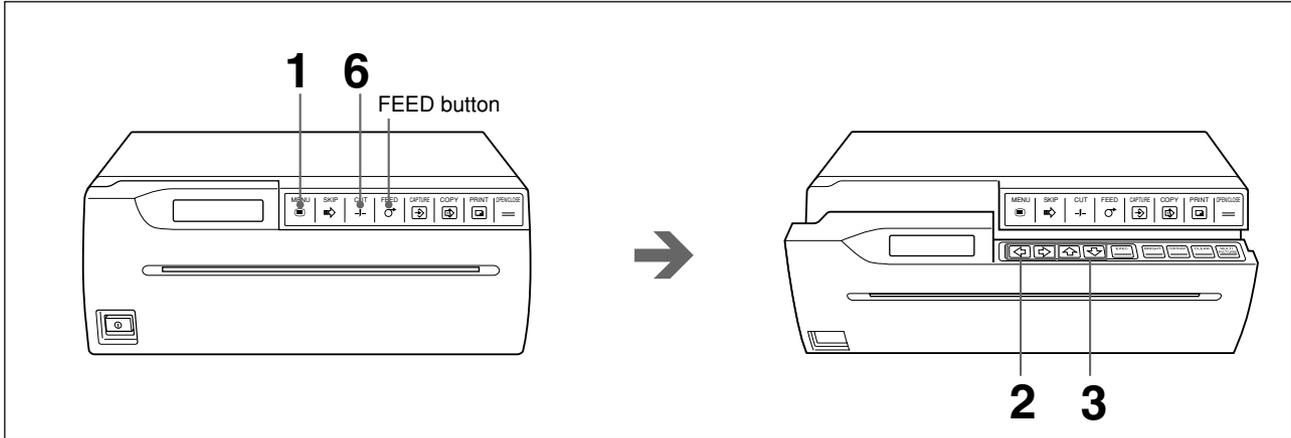

- 1** Press the MENU button.
The door panel opens slightly and the sub panel appears.
The main menu appears in the printer window display.
- 2** Display the PRINTER menu by pressing the \leftarrow or \rightarrow button.
- 3** Display the AUTO CUT sub menu by pressing the \uparrow or \downarrow button.

AUTO CUT
*ON / OFF

- 4** Set AUTO CUT to OFF on the AUTO CUT sub menu of the PRINTER main menu

AUTO CUT
ON / *OFF

The printer enters in the auto-cut off mode. The printer is in this mode until you change the auto cut mode setting.

- 5** Start printing.
Press the PRINT button to print the image or press the COPY button to print images stored in memory, and make the desired number of printouts.

Continue to next page →

- 6** Press the CUT button.
The paper is cut.

To add a margin

Before you press the CUT button in step 6, press and hold down the FEED button until the paper is fed out to the desired margin.

You can set a margin using the AUTO FEED sub menu. (See page 80.)


Note

Use the FEED button to feed the paper. Pulling on the paper without using the FEED button may result in mechanical problems.


Connection

Notes

- Turn off the power to each device before making connection.
- Connect the AC power cord last.
- Before making connections, see “Important safeguards/notices for use in the medical environment” on page 54.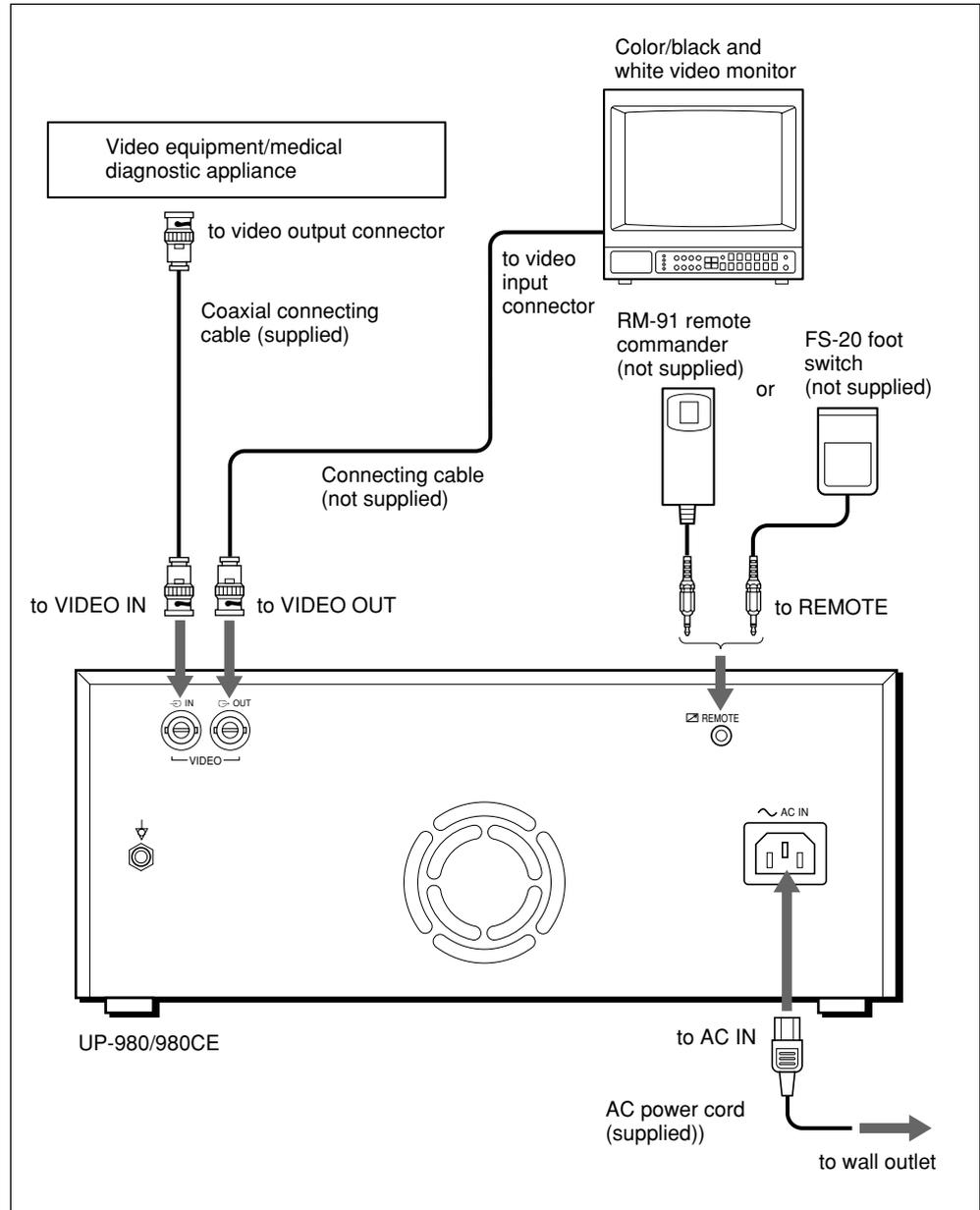

Menu operation procedure

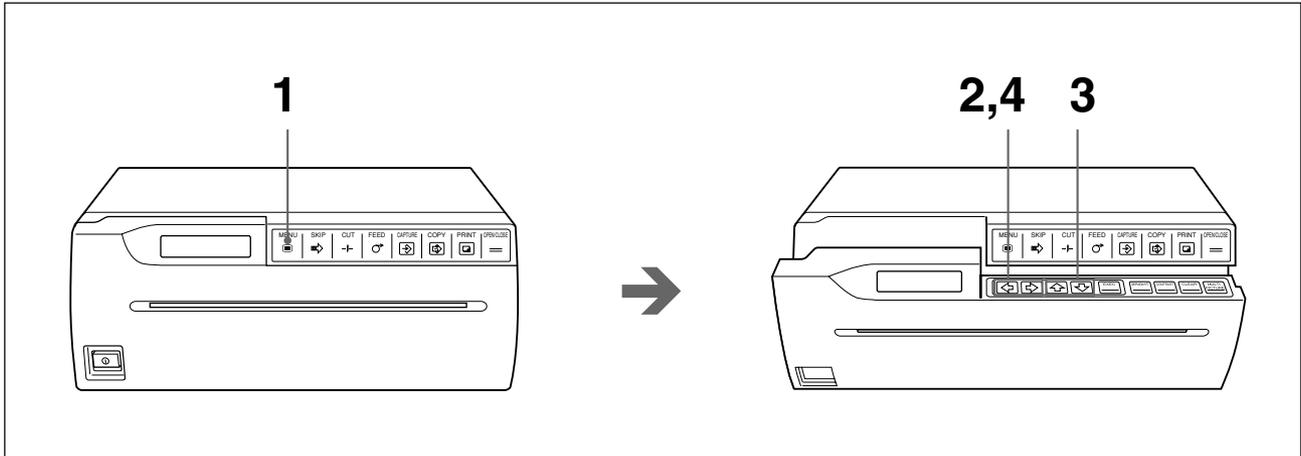- 1** Press the MENU button.
The door panel opens slightly and the sub panel appears.
The main menu appears in the printer window display.

MENU
*PICTURE / LAY / PRT

- 2** Display the desired main menu by pressing the ⇐ or ⇒ button.
- 3** Display the desired sub menu by pressing the ⇑ or ⇓ button.
- 4** Select the desired selection or change the value by pressing the ⇐ or ⇒ button.

To return to the regular window display

Press the MENU button to return to the main menu. Then press the MENU button again. The door panel closes and the printer window display returns to the regular display.

Or press the OPEN/CLOSE button. The door panel closes and the printer window display returns to the regular display.

Note

If you turn off the printer power with the door panel opened after you have changed the menu settings, those modified settings will become invalid. To execute and retain the menu settings, return to the regular window display by closing the door panel.

Setting Up the Printer (Continued)

PICTURE Menu

The PICTURE main menu consists of the following sub menus. These sub menus allow you to adjust the printouts and select the paper type.

PICTURE menu

MENU
*PICTURE / LAY / PRT

Sub menu (Item)	Contents of functions	Factory setting	Ref. page
PAPER TYPE	Selects the paper type.	2	76
GAMMA CURVE	Sets the desired printing tone.	1	76
IMAGE	Sets the printout to either positive printout or negative printout.	POSI	77
SHARPNESS	Obtains greater printout sharpness.	OFF	77
MEMORY	Selects either frame mode or field mode.	FRAME	77
BRIGHTNESS	Adjusts the printout brightness.	0	78
CONTRAST	Adjust the printout contrast.	0	78

PAPER TYPE

PAPER TYPE
TYPE *2 / 3

Selects the type of printing paper to be used.

TYPE 2: Thermal paper UPP-210SE/210HD

TYPE 3: Thermal film UPT-210BL


For details on the printing paper, see page 97.

GAMMA CURVE

GAMMA CURVE
TONE *1 / 2 / 3

Sets the printing tone to one of TONE 1, TONE 2 or TONE 3.

The diagram below shows the curve of each tone for your reference.

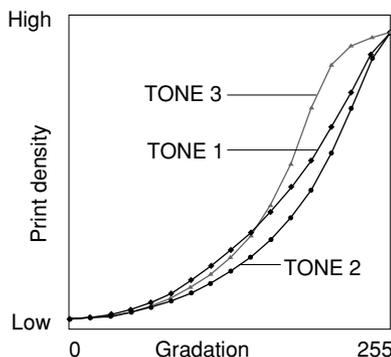IMAGE

IMAGE
*POS | / NEGA

Selects either positive or negative printouts.

POS: Normally make printouts using this setting.

NEGA: To make negative printouts, select NEGA.

SHARPNESS

SHARPNESS
ON / *OFF

Adjusts the printout sharpness.

ON: Obtains greater sharpness.

OFF: Normally make printouts using this setting.

Note

SHARPNESS is effective only when the color (NTSC or PAL) or black and white (EIA or CCIR) video signal is input.

MEMORY

MEMORY
*FRAME / FIELD

Sets the memory mode.

FRAME: Normally make printouts using this setting.

FIELD: When printing fast-moving pictures (such as a ball being thrown), the printout may blur. If this happens, use FIELD. The printout definition will be poorer, but less blurred.

Note

MEMORY is effective only when the interlace signal is input.

Setting Up the Printer (Continued)

BRIGHTNESS/CONTRAST

BRIGHTNESS sub menu

BR I G H T N E S S : 0
-----*-----

CONTRAST sub menu

C O N T R A S T 0
-----*-----

Both brightness and contrast settings are divided into 15 steps from -7 to +8.

BRIGHTNESS: Adjusts the brightness of the printouts.

The printout becomes brighter in the + direction by pressing the ⇨ button.

The printout becomes darker in the - direction by pressing the ⇩ button.

CONTRAST: Adjusts the contrast of the printout.

The contrast is increased in the + direction by pressing the ⇨ button.

The contrast is decreased in the - direction by pressing the ⇩ button.

Note

You cannot adjust the image once it has been captured in memory. Restore an image after adjustment.

Displaying the BRIGHTNESS/CONTRAST sub menus easily

Press the BRIGHT button on the sub panel to display the BRIGHTNESS sub menu easily and CONTRAST button to display the CONTRAST sub menu. (See pages 63 and 68.)

LAYOUT Menu

The LAYOUT main menu consists of the following sub menus. These sub menus allow you to adjust the printout size, printout area, and set printer operation functions frequently used in daily operations.

LAYOUT menu

MENU
PIC / *LAYOUT / PRT

Sub menu (item)	Contents of functions	Factory setting	Ref. page
IMAGE SIZE	Selects the printing direction in the vertical or horizontal direction.	STANDARD	79
AUTO FEED	Sets the amount of paper to be fed out in the auto cut mode.	0	80
FRAME COLOR	Selects the border color in the multi picture mode	WHITE	80
DIRECTION	Selects the printing direction from bottom or top.	NORMAL	80
MIRROR	Selects whether to rotate the image around its vertical axis when printing.	OFF	81
MULTIPICTURE	Sets the number of images to be printed on one page.	1	81
ASPECT	Selects the aspect ratio.	4:3	81
ASPECT ADJUST	Adjusts the selected aspect ratio.	00	82
H-SHIFT	Specifies the horizontal shift.	194 ¹⁾ 194 ²⁾	82
H-SIZE	Specifies the horizontal size	1200 ¹⁾ 1200 ²⁾	82
V-SHIFT	Specifies the vertical shift.	22 ¹⁾ 30 ²⁾	82
V-SIZE	Specifies the vertical size.	478 ¹⁾ 570 ²⁾	82

1) When the NTSC or EIA video signal is input:

2) When the PAL or CCIR video signal is input:


IMAGE SIZE

IMAGE SIZE
*STANDARD / SIDE

Selects the vertical or horizontal printing direction.

STANDARD: Prints in the vertical direction.

SIDE: Prints in the horizontal direction.

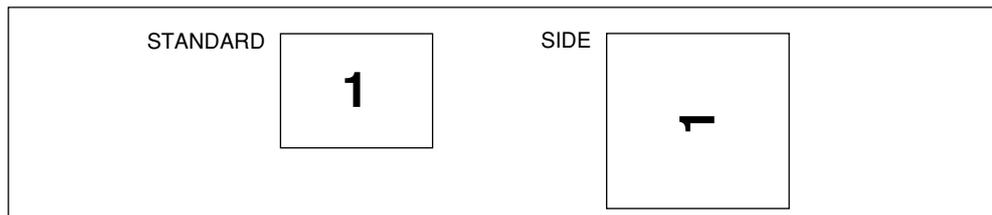Note

When you change the IMAGE SIZE setting, MULTIPICTURE is reset to 1 (in the single picture mode) if 2, 4 or 6 is selected on the MULTIPICTURE sub menu (in the multi picture mode).

Setting Up the Printer (Continued)

AUTO FEED

Sets the margin at the top and bottom of the printout. Used to set the paper size after printing.

The margin can be set in 15 steps from 0 to +15. One step corresponds to a 5 mm margin. Thus, up to 75 mm margin can be added to the top and bottom of the printout.

The amount of paper to be fed out is shortened by pressing the ⇐ button.
The amount of paper to be fed out lengthened by pressing the ⇒ button.

FRAME COLOR

Selects the border color of the reduced images in the multi picture mode.

BLACK: The border color is black.

WHITE: The border color is white.


DIRECTION

Selects whether the top or the bottom of the screen is to be printed first.

NORMAL: Normally make printouts using this setting. Printing is done from the bottom of the screen.

REVERSE: Starts printing from the top of the screen.

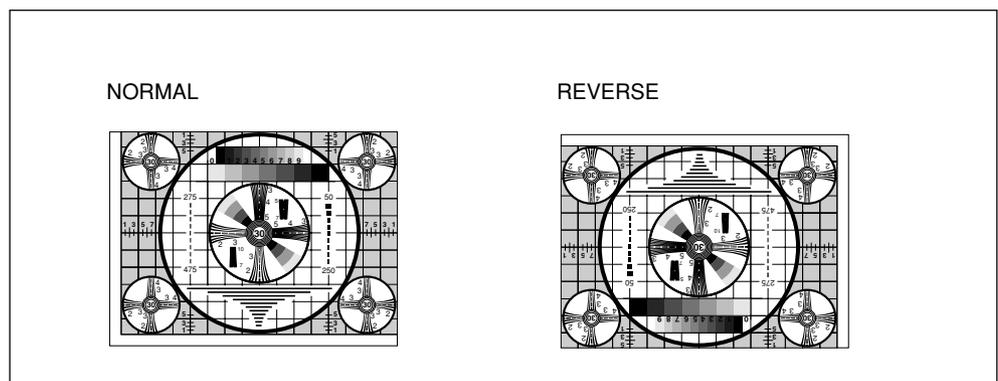MIRROR

Used to make a mirror image printout (the stored image is rotated around its vertical axis when printed).

OFF: Normally make printouts using this setting.

ON: Prints mirror image.

MULTIPICTURE

Sets the number of images to be printed on one page.

Item	Number of images
1	1 ¹⁾
2	2 ²⁾
4	4 ²⁾
6	6 ²⁾


1) The mode, where one full-size image is printed on one page, is called the single picture mode in this manual.

2) The mode, where multiple reduced images are printed on one page, is called the multiple picture mode in this manual.

Displaying the MULTIPICTURE sub menu easily

Press the MULTIPICTURE button on the sub panel.


ASPECT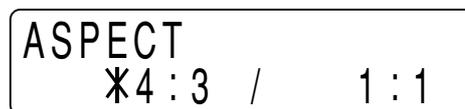

Selects the aspect ratio.

4 : 3: Normally make printouts using this setting.

1 : 1: When the aspect ratio of the video signal is 1:1, set to 1:1.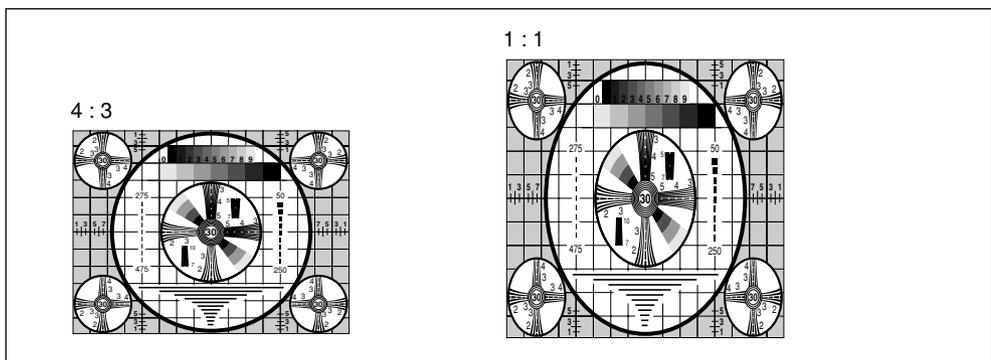

ASPECT ADJUST

ASPECT ADJUST
00

Makes fine adjustment to the selected aspect ratio.

To make a printout wider, increase the value by pressing the ⇨ button.

To make a printout taller, decrease the value by pressing the ⇩ button.

H-SHIFT/H-SIZE/V-SHIFT/V-SIZE

H-SHIFT sub menu

H-SHIFT:
194dots

H-SIZE sub menu

H-SIZE:
1200dots

V-SHIFT sub menu

V-SHIFT:
22lines

V-SIZE sub menu

V-SIZE:
478lines

In the AUTO SCAN mode, the printer automatically adjusts the printout size according to the input signals. However, a satisfactory printout may not be obtained even with this mode activated. There may be missing portions or black borders appearing on printouts. In such a case, you can make fine adjustments using the H-SHIFT, H-SIZE, V-SHIFT and V-SIZE sub menus of the LAYOUT menu.

For detailed information, see “Adjusting the Printout Size” on page 88.

PRINTER Menu

The PRINTER main menu consists of the following sub menus. These sub menus allow you to set the printer operation functions and specifications.

PRINT menu

MENU
PIC/LAY/*PRINTER

Sub menus (item)	Contents of functions	Factory setting	Ref. page
PRINT QTY	Sets the number of printouts.	1	83
LOAD	Loads the saved settings.	1	84
SAVE	Saves all menu settings	1	84
INITIALIZE	Resets all settings currently loaded to the factory settings.	—	84
INTERRUPT	Enables an image to be captured while printing.	OFF	84
BEEP	Selects whether the operation and error tones sound.	ON	85
LCD CONTRAST	Adjusts the contrast of the printer window display.	0	85
AGC	Adjusts the input signal level.	OFF	85
TRAP FILTER	Sets the trap filter to ON or OFF.	OFF	85
75 ohm	Sets the 75-ohm termination to ON or OFF.	ON	86
CLEAR	Clears the images stored in memory.	—	86
AUTO CUT	Selects whether the paper is cut automatically after printing.	ON	86
AUTO PRINT	Selects whether the printer starts printing automatically in the multi picture mode.	OFF	87
AUTO SCAN	Sets the printer to the auto scan mode.	0	87
SCAN INPUT	Sets the printer in the auto scan mode temporarily for the current input signal when AUTO SCAN is set to OFF.	—	87
PIXEL DENSITY	Selects the line density.	STANDARD	87

PRINT QTY

PRINT QTY Q1

Sets the number of printouts. You can set any number up to 10.

Setting Up the Printer (Continued)

LOAD/SAVE

LOAD sub menu

LOAD	<EXEC>
*1 /	2 / 3

SAVE sub menu

SAVE	<EXEC>
*1 /	2 / 3

You can save up to three settings. These settings are managed according to the LOAD number. You can load only on setting at a time.

For details, see “Saving the Menu Settings” on page 92.

INITIALIZE

INITIALIZE	<EXEC>
------------	--------

Resets the currently adjusted settings to the factory setting values.
To initialize, press the EXEC button.

INTERRUPT

INTERRUPT	
ON /	*OFF

Sets whether the PRINT and CAPTURE buttons are activated to enable interruption of the printing process and capture an image in memory.

Printer operation differs between the single picture and multi picture modes.

In the single picture mode:

ON: By pressing the PRINT button while printing, the printer stops printing, captures the image displayed, and starts printing immediately.

OFF: Even if you press the PRINT button, the printer continues printing without stopping.

In the multi picture mode:

ON: By pressing the CAPTURE button while printing, the printer stops printing, captures the new image and displays it at the position where the cursor has been placed.

OFF: Even if you press the CAPTURE button, the printer continues printing without stopping.

When remotely controlling the printer

The button on the remote control unit or foot switch functions exactly same as the PRINT button and CAPTURE button.

For details, see pages 63 and 68.

BEEP

BEEP
*ON / OFF

Selects whether the operation and error tones sound whenever you press a button.

ON: Enables the tones.

OFF: Disables the tones.

LCD CONTRAST

LCD CONTRAST 0
-----*-----

Adjusts the contrast of the printer window display.

The number decreases and the contrast weakens by pressing the ⇐ button.

The number increases and the contrast strengthens by pressing the ⇒ buttons.

AGC (Automatic gain control)

AGC
ON / *OFF

Adjusts the input signal to the optimum printing level.

OFF: Normally make printouts using this setting (when the proper signal is input).

ON: When the printout image appears blackish or whitish, select this position to adjust the input signal to the optimum level.

Note

AGC is effective only when the color (NTSC or PAL) or black and white (EIA or CCIR) video signal is input.

TRAP FILTER

TRAP FILTER
ON / *OFF

Sets the trap filter to ON or OFF in accordance with the input signal.

ON: When the input signal to be printed is in color.

OFF: When the input signal to be printed is in black and white.

Note

TRAP FILTER is effective only when the color (NTSC or PAL) or black and white (EIA or CCIR) video signal is input.

Setting Up the Printer (Continued)

75 ohm

75 ohm
*ON / OFF

Selects whether the printer is terminated with 75-ohm impedance.

ON: When no equipment is connected to the VIDEO OUT connector of the printer.

OFF: When equipment such as a monitor or a VTR is connected to the VIDEO OUT connector of the printer.

When you connect two printers to one video equipment, set the 75 ohm sub menu of one of the printer to ON, and the other to OFF.

CLEAR

CLEAR
<EXEC>

Clears all the images stored in memory in the multi picture mode.
To clear the images, press the EXEC button.

Display the CLEAR sub menu easily

Press the CLEAR button on the sub panel. (See page 69.)

AUTO CUT

AUTO CUT
*ON / OFF

Sets the printer to the auto cut mode. The paper is automatically cut when printing is completed.

ON: Sets the printer to auto cut mode.

OFF: The paper should be cut manually by using the CUT button. Using this mode, you can save paper and make more printouts per roll of paper.

AUTO PRINT

AUTO PRINT
ON / *OFF

Selects whether the printer starts printing automatically in the multi picture mode.

ON: When the determined number of images have been captured, the printer starts printing automatically without the COPY button being pressed.

OFF: You have to press the COPY to start printing in the multi picture mode.

AUTO SCAN

AUTO SCAN
*ON / OFF

Selects the auto scan mode.

ON: Selects the auto scan mode. The printer adjusts the items related to printout size in accordance with the input signal.

OFF: When the AUTO SCAN mode is switched from ON to OFF, settings of items automatically adjusted in accordance with the input signal when AUTO SCAN was set to ON are retained. OFF is recommended when the signal that may not be correctly synchronized, such as the signal from VTR, is input.

SCAN INPUT

SCAN INPUT
<EXEC>

When the EXEC button in this sub menu is pressed, the printer scans for the signal currently input and resets the following items to the factory settings for the scanned signal.

When you change the input signal, be sure to press the EXEC button in this sub menu.

Items reset to the factory settings: IMAGE SIZE, MULTI PICTURE, ASPECT ADJUST, H-SHIFT, H-SIZE, V-SHIFT and V-SIZE.

PIXEL DENSITY

PIXEL DENSITY
*STANDARD / HIGH

Selects the line density.

STANDARD: Normally make printouts using this setting.

HIGH: Sets the print line density to high density and obtains the better print quality. However, the printing speed is slower than in the STANDARD mode.

Adjusting the Printout Size

In the auto scan mode, the printer automatically adjust the print area (printout size) according to the type of the input signal. With certain types of input signals, however, satisfactory printouts may not be obtained even with the auto scan mode activated. If there are missing portions or black borders on the printout, fine adjustments on the H-SHIFT, H-SIZE, V-SHIFT and V-SIZE sub menus are required.

This section explains the content of the adjustable items and how to adjust the printout size.


Adjustable items

The following lists show the items you can adjust using the H-SHIFT, H-SIZE, V-SHIFT and V-SIZE sub menus.

Sub menu	Contents of adjustment
H-SHIFT	Specifies the horizontal shift.
H-SIZE	Specifies the horizontal size.
V-SHIFT	Specifies the vertical shift.
V-SIZE	Specifies the vertical size.

Relation between the print area and input image

The relation between the print area and input image is shown in the example below.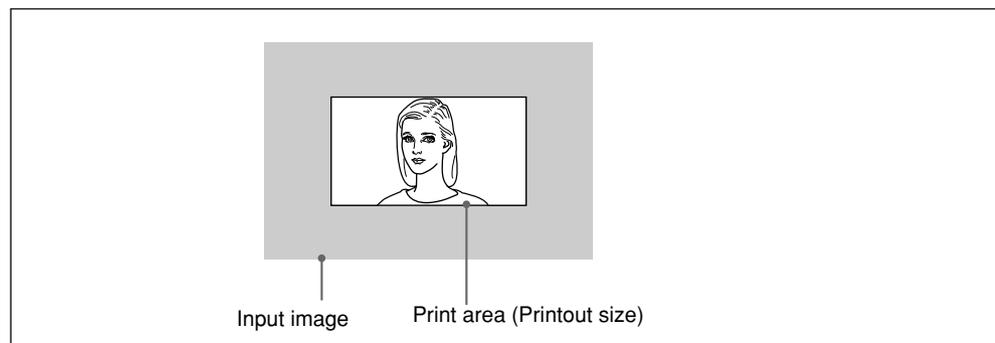

The relation between the print area and input image is automatically adjusted in auto scan mode. However, you can make fine adjustment if satisfactory printouts cannot be obtained.

Adjustment of the position relation between the print area and input image

You can adjust the position relation using the H-SHIFT and V-SHIFT sub menus.

H-SHIFT: Shifts the print area to the right or left in the input image.

V-SHIFT: Shifts the print area upwards or downwards in the input image.

Adjustment of the print area (printout size)

You can adjust the print area using the H-SIZE and V-SIZE menus.

H-SIZE: With the left edge remaining fixed, the print size is widened or narrowed.

V-SIZE: With top edge remaining fixed, the print area is enlarged downwards or reduced.

Adjusting the print area (printout size)

This subsection explains how to adjust the print area using the example.

Note

The values adjusted on each sub menu remain even if the printer is turned off.

The following examples of the printer window display indicate the values automatically adjusted in auto scan mode when the NTSC signal is input.


Adjustment on the H-SHIFT sub menu

Specifies the horizontal shift value in dots.

The following printer window display shows the value set in auto scan mode.

H-SHIFT:
194dots

Example: When the left portion of the printout is missing with above setting; Reduce the H-SHIFT value.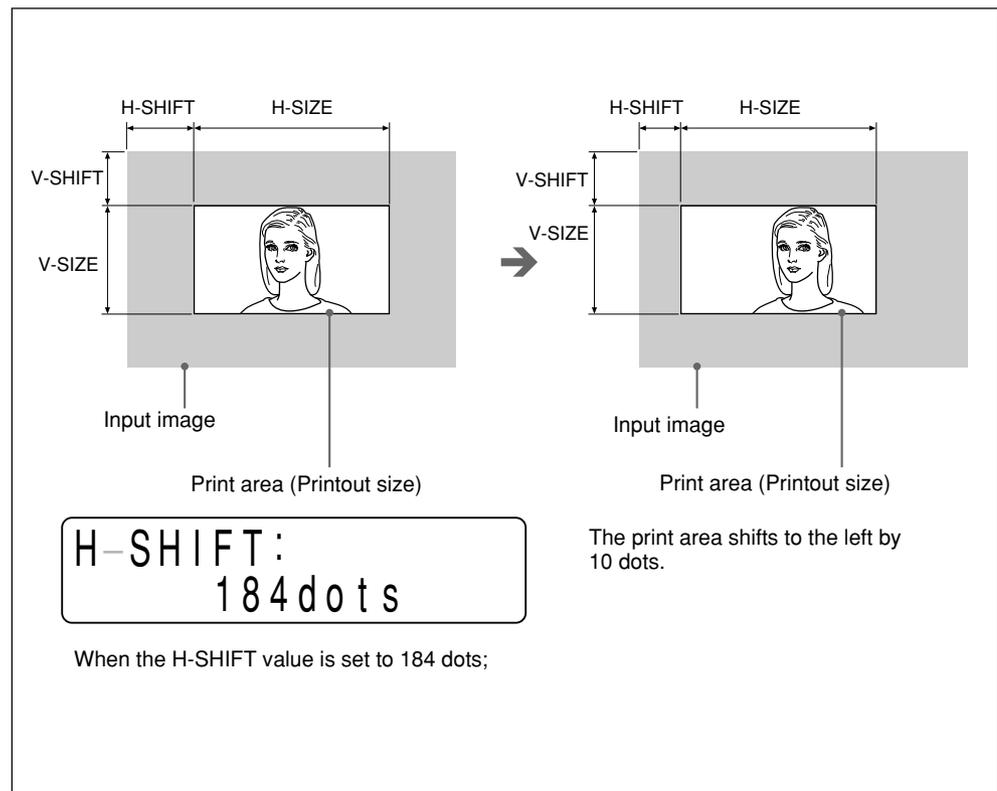

Setting Up the Printer (Continued)

Adjustment on the H-SIZE sub menu


Specifies the horizontal size in dots.

The following printer window display shows the value set in auto scan mode.

```
H-SIZE:  
1200dots
```

Example: The horizontal size is too small with the above setting, and the right portion of the picture will be missing if the horizontal shift is adjusted.

Increase the H-SIZE value.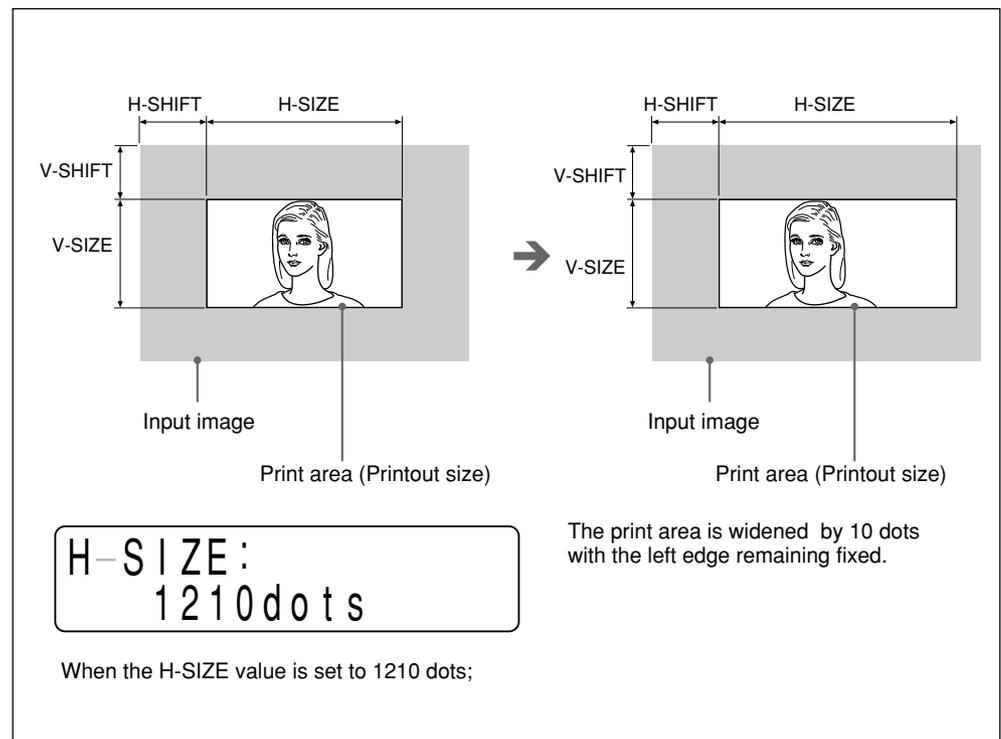


Adjustment on the V-SHIFT sub menu

Specifies the vertical shift value in lines.

The following printer window display shows the value set in auto scan mode.

```
V-SHIFT:  
22lines
```

Example: A black border appears on the lower part of the printout with the above setting.
Decrease the V-SHIFT value.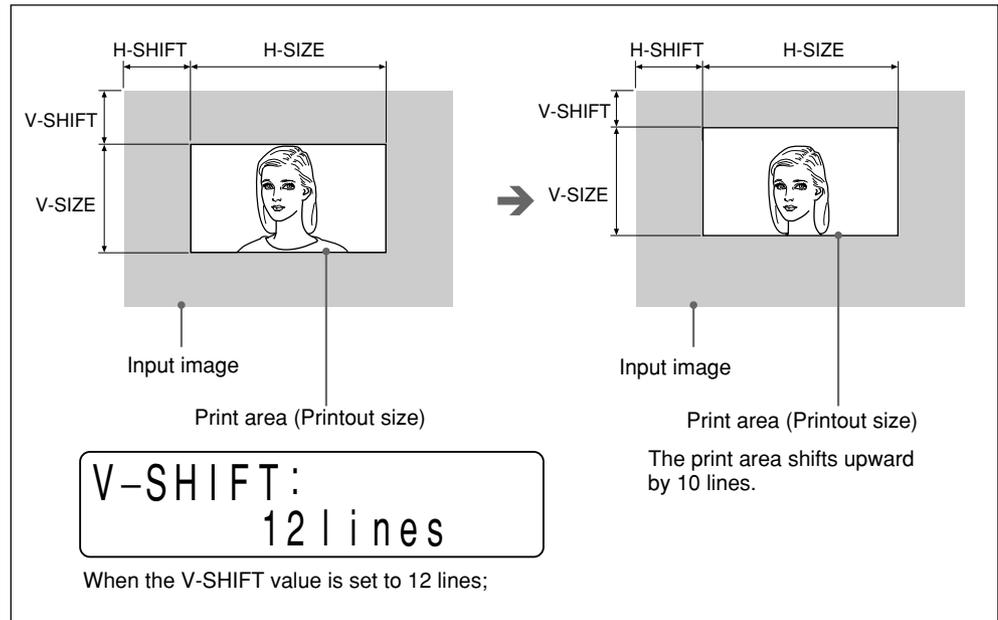


Adjustment on the V-SIZE sub menu

Specifies the vertical size in lines.

The following printer window display shows the value set in auto scan mode.

V-SIZE:
478 lines

Example: The vertical size is too large with the above setting, and a black border appears on the lower part of the printout.
Decrease the V-SIZE value.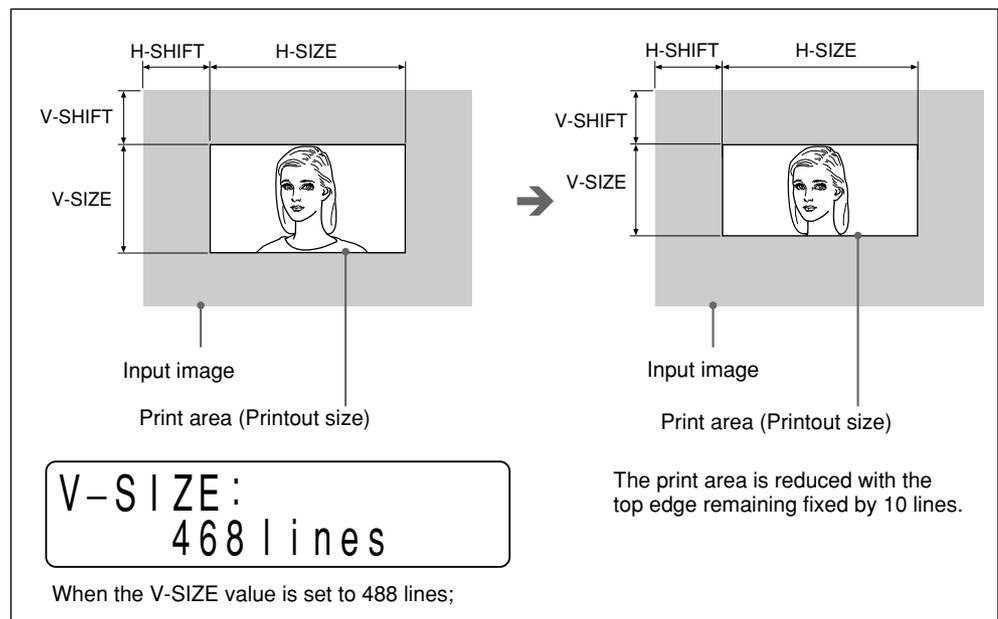

Saving the Menu Settings

You can save any settings you have made on all sub menus and load the desired settings. The printer operates according to the loaded settings.

You can modify the loaded settings and the printer will then operate according to the modified settings. These modified settings are retained even if the printer power is turned off.

Up to three different settings can be saved. All settings are retained even if the printer power is turned off.

Note

At the factory, factory setting values have been set to all of sub menus.

For the factory setting values, see “PICTURE Menu” on page 76, “LAYOUT Menu” on page 79 and “PRINTER Menu” on page 83.

For the H-SHIFT, H-SIZE, V-SHIFT and V-SIZE sub menus of the LAYOUT menu, factory settings for the NTSC color video signal and EIA black and white video signal have been set.

Saving the settings

Note

Be sure that all settings have been made.

- 1** Press the MENU button.
The door panel opens slightly and the sub panel appears.
The main menu appears in the printer window display.
- 2** Display the PRINTER menu by pressing the \leftarrow or \rightarrow button.
- 3** Display the SAVE sub menu by pressing the \updownarrow or \downarrow button.
- 4** Select the desired SAVE number by pressing the \leftarrow or \rightarrow button.
- 5** Press the EXEC button.
The settings are saved with the SAVE number selected in step 4.

Loading the saved settings

The printer operates according to the loaded settings.

- 1** Press the MENU button.
The door panel opens slightly and the sub panel appears.
The main menu appears in the printer window display.
- 2** Display the PRINTER menu by pressing the \leftarrow or \rightarrow button.
- 3** Display the LOAD sub menu by pressing the \updownarrow or \downarrow button.
- 4** Select the desired LOAD number by pressing the \leftarrow or \rightarrow button.

-
- 5** Press the EXEC button.
The settings with the LOAD number selected in step 4 are loaded.

To copy the settings made for one certain LOAD number to another SAVE number

- 1** Display the LOAD sub menu and select the LOAD number for which settings to be copied are stored by pressing the ⇐ or ⇒ button.
- 2** Press the EXEC button.
- 3** Display the SAVE sub menu and select the target SAVE number by pressing the ⇐ or ⇒ button.
- 4** Press the EXEC button.

To modify settings loaded in step 1 and save them

Modify the desired settings after step 2, then save the modified settings in step 3.

Precautions

On the safety

- Check the operating voltage before operation.
Operate the unit only with a power source specified in the “Specifications”.
- Stop operation immediately if any liquid or solid objects fall into the cabinet.
Unplug the unit and have it checked by qualified personnel.
- Unplug the unit from a wall outlet if you will not be using it for a long time.
Disconnect the power cord by grasping the plug. Never pull the cord itself.
- Do not disassemble the cabinet. Refer servicing to qualified personnel only.
- Keep fingers clear of the door panel when the the door panel is closing.
- Connect the power plug of the printer to a wall outlet that is safely grounded.

On operation

Do not turn the power off while the printer is printing. The thermal head may be damaged.

On printer carriage

Do not carry or move the printer when a paper roll is being placed in the printer; Doing so may cause a malfunction.

On installation

- Place the printer on a level and stable surface during operation.
- Do not install the printer near heat sources. Avoid locations near radiators or air ducts. Also, do not place subject to direct sunlight or excessive dust, humidity, mechanical shock or vibration.
- Provide adequate air circulation to prevent heat buildup. Do not place the printer on surfaces such as rugs, blankets, etc., or near materials such as curtains and draperies.
- If the printer is subjected to wide and sudden changes in temperature, such as when it is moved from a cold room to a warm room or when it is left in a room with a heater that tends to produce large amounts of moisture, condensation may form inside the printer. In such cases the printer will probably not work properly, and may even develop a fault if you persist in using it. If moisture condensation forms, turn off the power and allow the printer to stand for at least one hour.

Maintenance

Cleaning the cabinet

Do not use strong solvents to clean the printer. Thinner or abrasive cleansers will damage the cabinet.


Cleaning the thermal head

If the printout is dirty or white stripes appear on the printouts, clean the thermal head using the supplied cleaning sheet.


If the horizontal stripes appear on the printouts when the UPT-210BL is used, clean the thermal head.

Also, when the UPT-210BL is used for a long time, the vertical stripes may appear on the printouts. This results in low density printouts. In such case, clean the thermal head.

- 1 Press the power ON/OFF switch to turn on the printer.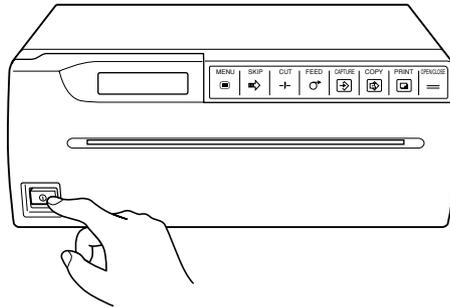


- 2 Press the OPEN/CLOSE button to open the door panel.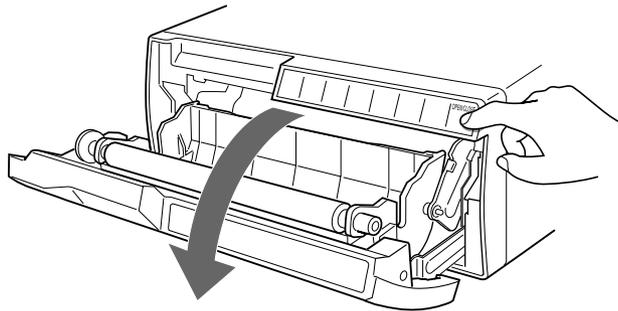

Continue to next page →

Precautions (Continued)

- 3 Place the cleaning sheet on the tray, and insert the end of the cleaning seat with the black surface facing down, into the paper insertion slot and feed the paper with your hands until the end comes out from the paper outlet.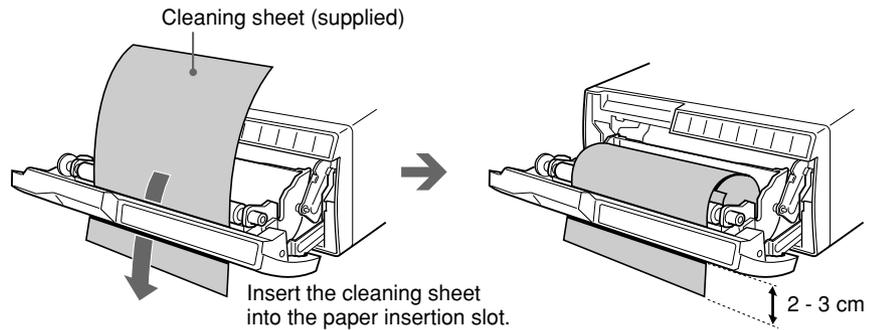

- 4 Press the OPEN/CLOSE button and keep it pressed. The door panel closes and the printer starts cleaning the head. When the buzzer sounds and the printer starts ejecting the cleaning sheet, release the OPEN/CLOSE button. The following message is displayed while cleaning.

HEAD CLEANING

- 5 Press the OPEN/CLOSE button to open the door panel when the head cleaning is completed and the printer stops ejecting the cleaning sheet, then remove the cleaning sheet.

Notes

- Do not press the PRINT or COPY button while the cleaning sheet is in the printer.
- Clean the head only when necessary. If you clean the head too often, it may cause a malfunction.

Type of paper

- Use only paper specified for this printer. The use of other paper may result in reduced printer performance and poor print quality.
- The following types of paper are available.
 - UPP-210SE (Normal)
 - UPP-210HD (High density)
 - UPT-210BL (Thermal film)
- Before making a printout, make sure that the paper is suitable to your printer and set the paper type corresponding to your paper on the PAPER TYPE sub menu of the PICTURE main menu. (See page 76.)

Type of paper	Paper
TYPE 2	UPP-210SE/210HD
TYPE 3	UPT-210BL

Storing paper

- Store unused or printed paper or film in a cool, dark place (below 30°C or 86°F). We recommend that you store printed paper or film in a polypropylene pouch in the photo album.
- Do not store unused or printed paper or film in hot or humid place.
- Do not leave unused or printed paper or film in direct sunlight or other bright places for extended periods.
- Do not allow any volatile organic solvent or vinyl chloride to touch the printed paper or film. Alcohol, plastic tape or film will cause the printout to fade.
- To attach the printed paper to another piece of paper, use double-sided adhesive tape, or water-based or solid glue. (UPP-210SE/210HD only)
- Do not stack printed paper or film on or under a diazo copy sheet. The printout may become blackened.

Specifications

Thermal head

Thin-film thermal head (with built-in drive IC) 1280 dots

Gradation

256

Picture elements

EIA: 1280 × 506 dots

CCIR: 1280 × 610 dots

Print size (at factory setting)

STANDARD mode

EIA: 187 × 140 mm

CCIR: 187 × 138 mm

SIDE mode

EIA: 249 × 188 mm

CCIR: 249 × 186 mm

Printing speed

EIA: About 10 seconds/screen (aspect ratio 4:3)

CCIR: About 12 seconds/screen (aspect ratio 4:3)

Picture memory

2048 × 2048 × 8 bits

Input/output connectors

VIDEO IN (BNC)


- EIA or CCIR Composite video signals
1.0 Vp-p, 75 ohms/high-impedance switching (EIA/CCIR automatically discriminated)

- Hi-scan signal (31.5 kHz)

VIDEO OUT (BNC)

EIA or CCIR Composite video signals
1.0 Vp-p, 75 ohms, loop-through

REMOTE (stereo minijack)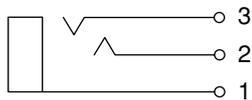

1 GND

2 PRINT SIGNAL (TTL)

When the LOW pulse over 100 msec is input, printing starts.

3 PRINT BUSY (TTL)

Goes HIGH during printing.

Power requirements and consumption

120 V AC, 50/60 Hz, 2.4 A

220 to 240 V AC, 50/60 Hz, 1.3 A

Operating temperature

5°C to 35°C (41°F to 95°F)

Operating humidity

20 % to 80 % (non condensation allowed)

Storage and transport temperature

-20°C to 60°C (-4°F to 140°F)

Storage and transport humidity

20 % to 90 % (non condensation allowed)

Dimensions

Approx. 316 × 132 × 305 mm (w/h/d)
(12½ × 5¼ × 12⅛ inches)

Mass

Approx. 8 kg (17 lb 10 oz), Main unit only

Protection against electric shock:

Class I

Protection against harmful ingress of water:

Ordinary

Degree of safety in the presence of flammable anesthetics or oxygen:

Not suitable for use in the presence of flammable anesthetics or oxygen

Mode of operation:

Continuous

Supplied accessories

Paper roll (UPP-210HD) (1)

BNC - BNC connecting cable (1)

AC power cord (1)

Head cleaning sheet (1)

Instructions for Use (1)

Design and specifications are subject to change without notice.

Troubleshooting

The following troubleshooting checks will help you correct the most common problems you may encounter with your printer. Before proceeding with these trouble check, first check that the power cord is firmly connected. should the problem persist, unplug the printer and contact your Sony dealer or local authorized Sony service facility.

Symptom	Cause/remedy
White specks on first few printouts.	When printing with a newly inserted roll of paper, dust on the surface of the paper may cause white specks on the printouts. → Feed the paper by pressing the FEED button until clean paper appears, then cut by pressing the CUT button. (page 60)
Printing does not start when you press the PRINT or COPY button.	<ul style="list-style-type: none"> • Paper does not feed. <ul style="list-style-type: none"> → Is the paper slack? → Is the power turned on? → Are all connections correct? (page 73) → If the unit is in the multi picture mode, press the COPY button for printing. (page 67) • When the alarm buzzer sounds: <ul style="list-style-type: none"> → Has the thermal head overheated? (page 100) → Is the signal of the picture input? → Is the paper loaded correctly? (page 58) • Paper feeds, but printing does not start. <ul style="list-style-type: none"> → Is the paper loaded with the thermo-sensitive side up? (page 59)
Black borders or missing portions around the printout.	This may result according to the video signal input to the printer. → Adjust the printout size. (page 88)
Paper jam	<ul style="list-style-type: none"> • Open the door panel by pressing the OPEN/CLOSE button, then slowly pull out the jammed paper and remove it. • There is condensation within the unit. <ul style="list-style-type: none"> → Moving the unit suddenly from a cold place to a warm place often results in condensation forming. In the event of condensation forming, remove the paper, turn off the power and leave the unit for about one to two hours.
Printout is dirty.	The thermal head is dirty. → Clean the thermal head with the supplied head cleaning sheet. (page 95)
Horizontal stripes appear on the printouts when the UPT-210BL is used.	Did you clean the thermal head after changing paper on the UPT-210BL? → Clean the thermal head using the supplied cleaning sheet. (page 95)
Vertical stripes appear and printout density becomes low when the UPT-210BL is used.	Do you use the UPT-210BL for a long time? → Clean the thermal head using the supplied cleaning sheet. (page 95)
The printer stops printing when it prints continuously black images.	This is likely to occur if the printer prints continuously 15 or more dark pictures. In such a case, the buzzer sounds. This is because there is a protective circuit that guards against heat buildup of the thermal head. Stop printing for a while.
White lines or small letters on the screen are not printed clearly.	Is TRAP FILTER of the TRAP FILTER sub menu set to ON when the input signal is a black and white signal? (page 85)
Small squares appear over the whole screen.	Is TRAP FILTER of the TRAP FILTER sub menu set to OFF when the input signal is a color signal? (page 85)
The printout is too dark or too light.	<ul style="list-style-type: none"> • Is the 75 ohm sub menu set correctly? (page 86) • is PAPER TYPE of the PAPER TYPE sub menu set correctly? (page 76) • Is GAMMA CURVE of the GAMMA CURVE sub menu set correctly? (page 76)
The printout seems stretched.	Is ASPECT of the ASPECT sub menu set to 1:1? → Set to 4:3. (page 81)

Error Messages

If a problem occurs, an error message stating the problem appears in the printer window display. This section lists error messages together with their possible causes and remedies. Note the messages and act accordingly.


Error message	Possible cause and remedies
MOTOR TROUBLE	Motor trouble has occurred. → Contact your Sony service facility or Sony dealer.
SENSOR TROUBLE	Sensor trouble has occurred. → Contact your Sony service facility or Sony dealer.
CHECK PAPER	The paper roll has been exhausted. Or the paper is not loaded correctly. → Load the paper correctly. (page 58)
NO INPUT	The printer is not receiving an input signal. → Check that the connections between signal source equipment and the printer are secure. (page 73) → Check whether the video equipment is outputting a video signal in playback mode.
PLEASE WAIT	Please wait a little while.
PLEASE WAIT HEAD IN COOLING	The thermal head has overheated. → Leave the printer until the head cools down and this message disappears.
DOOR OPEN	The door panel is open. → Close the door panel.
INPUT MISMATCH	A different signal ¹⁾ is input. → Check the input signal.
OVER SPEC	The signal of the horizontal scanning frequency and the number of lines over the specified values is input. → Check the input signal.
NO IMAGE	The COPY button is pressed when no image is captured. Press the PRINT button to capture the image in the single picture mode. (page 61) Press the CAPTURE button to capture the image in the multi picture mode. (page 64)

- 1) The different signal has two kinds of meanings.
One — the signal is different from the currently selected signal. For example, when the settings of LOAD 1 are loaded, the signal specification saved in LOAD 1 is different from the previous signal.
The other meaning is when the signal whose horizontal scanning frequency and the number of lines are different from those of the current signal is input, with AUTO SCAN set to OFF.

Location and Function of Parts and Controls

For details, refer to the pages indicated in parentheses.


Front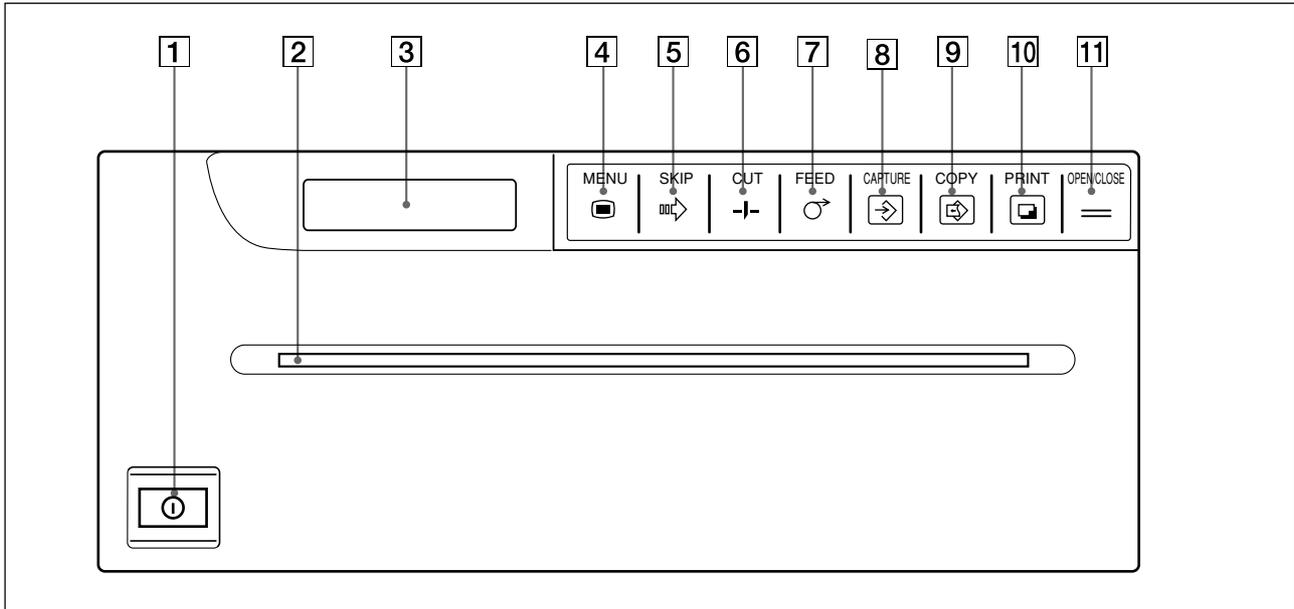

1 **Ⓞ Power ON/OFF switch (58, 61, 95)**
Turns the power on.

2 **Paper outlet (59, 96)**

3 **Printer window display**
Displays the printer condition in the regular print mode.
Displays the menu in menu operation mode.
If an error occurs, a corresponding error message is displayed.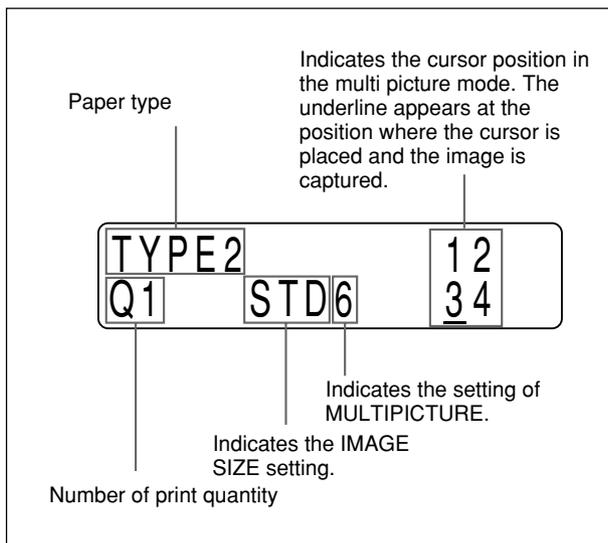

4 **☐ MENU button (64, 69, 71, 74, 75)**
Used for menu operation. By pressing the MENU button, the door panel opens slightly and the sub panel appears. Also, press the MENU button to return to the main menu from the sub menus.

5 **⏪ SKIP button (67)**
Press to move the cursor to the desired position in the multi picture mode.

6 **⏸ CUT button (60, 62, 68, 71)**
Press to cut the paper. Pressing this button during printing results in stopping the printing.

7 **⏩ FEED button (60, 62, 68, 72)**
Press to feed the paper. The paper is fed as long as the FEED button is held down. Pressing this button during printing results in stopping the printing.

8 **📷 CAPTURE button (67)**
Press to capture the images in memory in the multi picture mode. The button is not activated in the single picture mode.

Location and Function of Parts and Controls (Continued)

9  **COPY button (62, 67, 68)**

Press to print another copy of the previous printout in the single picture mode.
Press to print images stored in memory in the multi picture mode.


10  **PRINT button (62)**

Press to capture the image currently displayed on the video monitor and then start printing in the single picture mode. This button is not activated in multi picture mode

11 **OPEN/CLOSE button (58, 60, 62, 68, 95, 96)**

Press to open or close the front door. Also, press to interrupt printing.

Sub Panel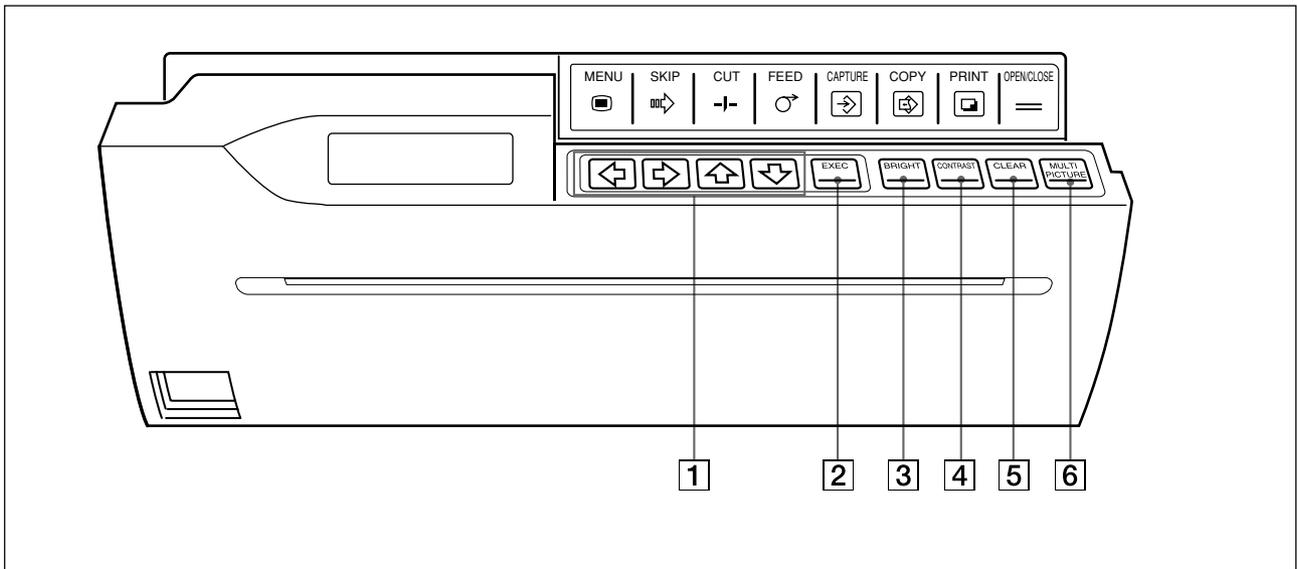

1 **Cursor keys (64, 69, 71, 74, 75)**

Used for menu operation.

2 **EXEC button (69, 74, 84, 86, 87, 92, 93)**

Press to execute settings on the sub menus.

3 **BRIGHT button (63, 68, 78)**

Press to display the BRIGHTNESS sub menu.

4 **CONTRAST button (63, 68, 78)**

Press to display the CONTRAST sub menu.


5 **CLEAR button (69, 86)**

Press to display the CLEAR sub menu in the multi picture mode.

6 **MULTI PICTURE button (65, 81)**

Press to display the MULTIPICTURE sub menu.

Back

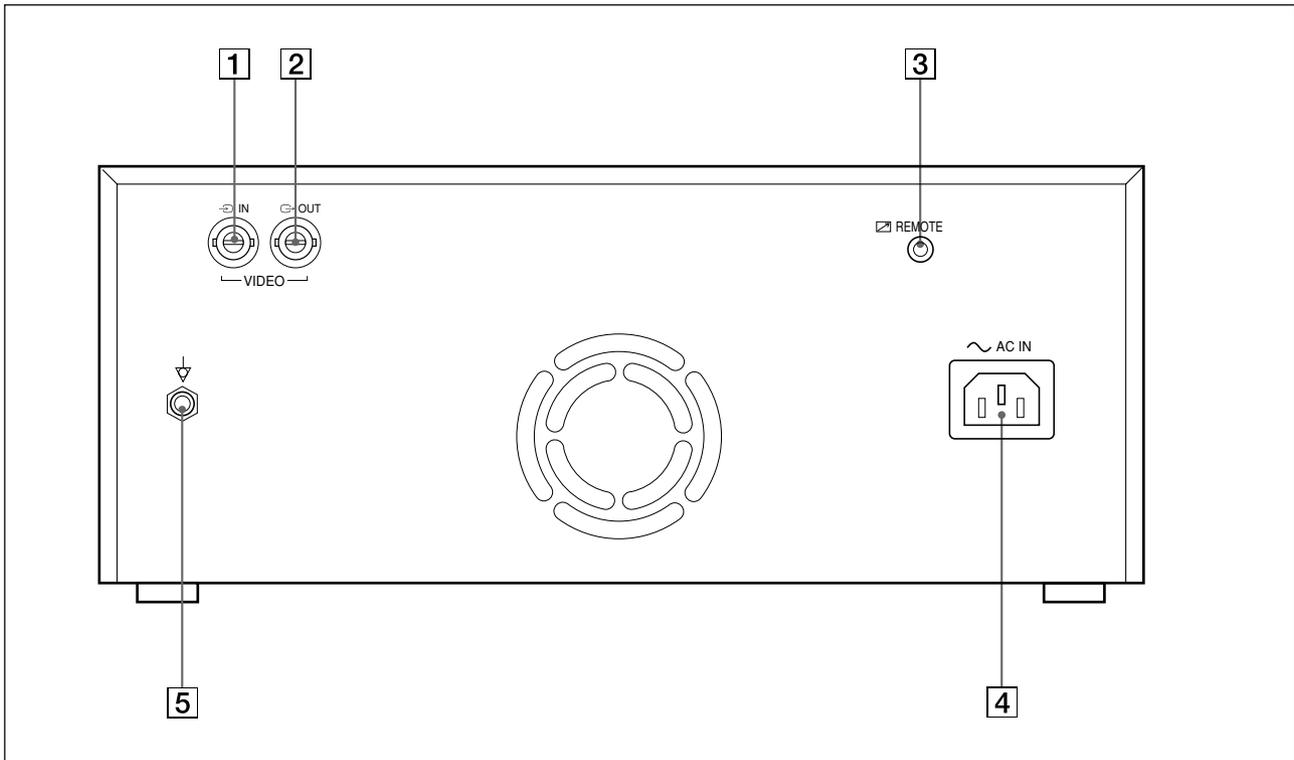1 → **VIDEO IN (input) connector (BNC type) (73)**

Connect to the video output connector of the video equipment.
Refer to “Important safeguards/notice for use in the medical environments” on page 54.

2 ↶ **VIDEO OUT (output) connector (BNC type) (73)**

Connect to the video input connector of the video monitor. The signal which is input to the VIDEO IN connector is output (through signal).
Refer to “Important safeguards/notice for use in the medical environments” on page 54.

3 ☑ **REMOTE connector (73, 98)**

Connect the optional RM-91 remote commander or the optional FS-20 foot switch for controlling print operation from a distance.

4 ~ **AC IN (AC power input) connector (73)**

Connect to a wall outlet using the supplied AC power cord.

5 ∇ **Equipotential terminal (73)**

Used to connect to the equipotential plug to bring the various parts of a system to the same potential.
Refer to “Important safeguards/notice for use in the medical environments” on page 54.

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Sony Corporation Printed in Japan