STORZ

KARL STORZ — ENDOSKOPE

KARL STORZ AIDA™ DVD-M

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Model 20204520-140 Basic Unit with integrated Smartscreen™ Touch Screen

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Model 20204520-140 Basic Unit with
integrated Smartscreen™ Touch Screen

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Important information for users of KARL STORZ equipment and instruments

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Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern, high-quality item of equipment from KARL STORZ.

This instruction manual is intended to serve as an aid in the proper setup, installation, and operation of the digital video, still image and audio capture device AIDA DVD-M. All essential details of the equipment and all actions required on your part are clearly presented and explained. We thus ask that you read this manual carefully before proceeding to work with the equipment. Keep this manual available for ready reference in a convenient and conspicuous location near the equipment.

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General warning

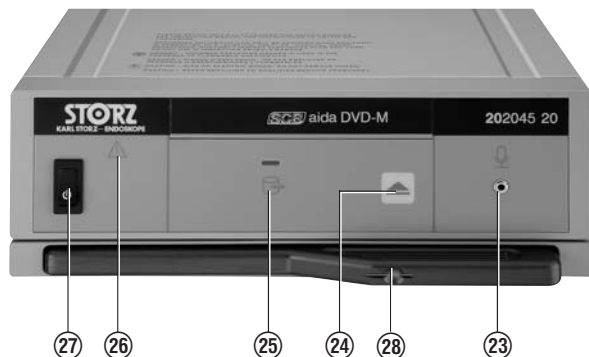
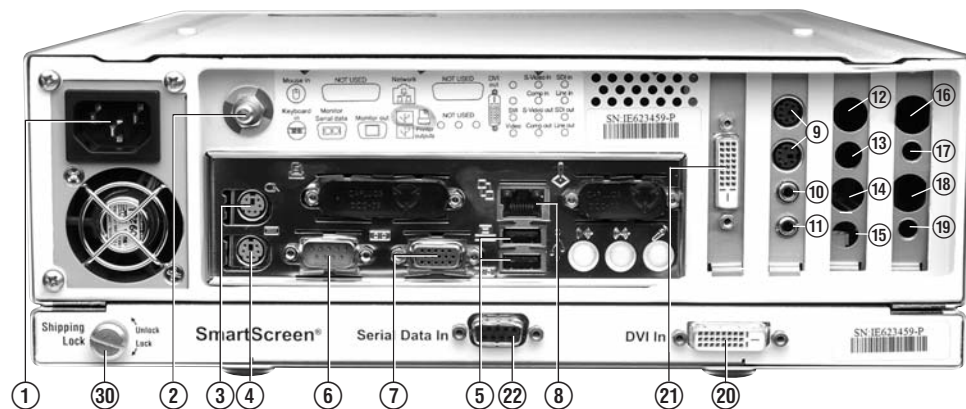
Users of KARL STORZ devices should be thoroughly trained in their use and the applicable medical procedures. Use of these devices and accessories should not be undertaken until the user has fully familiarized himself/herself with the instructions for use, assembly and care. Instruction manuals should be carefully studied and be available to the endoscopic team during the procedure; it is essential to follow the instructions contained in the instruction manual, paying particular attention to the Cautions and Warnings.

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
Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

20204520-140 Grundgerät mit integriertem SmartScreen™ Touch Screen
 20204520-140 Basic Unit with integrated SmartScreen™ Touch Screen
 20204520-140 20 204501-140 Базовый прибор с встроенным сенсорным смарт-экраном™

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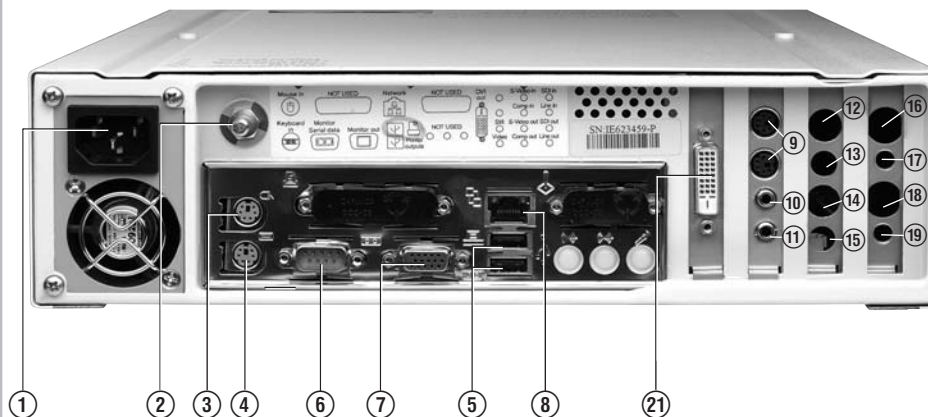Controls, displays, connectors, and their uses

- ① Power cord
- ② Grounding terminal
- ③ Mouse Input: for connecting a standard computer mouse
- ④ Keyboard Input: for a standard keyboard for diagnostic use
- ⑤ USB: upper socket for connecting a printer or other USB devices, lower socket for connecting a USB-device (storage)
- ⑥ Monitor Serial Data: connection for touch screen monitor (standard RS-232)
- ⑦ Monitor Out: VGA for connecting other devices (e.g. remote touch screen)
- ⑧ Network Connection: network interface RJ45
- ⑨ SCB Port (2)
- ⑩ ACC1 (Still): interface to KARL STORZ Camera Control Units for control of still image capture via camera buttons
- ⑪ ACC2 (Video): interface to KARL STORZ Camera Control Units for control of video via camera buttons
- ⑫ S-Video Input: connection for external S-Video [Y/C] source
- ⑬ Composite Video Input: interface for external composite video sources (composite cinch adaptor ⑬a is required)
- ⑬ a Composite-Cinch-Adapter
- ⑭ S-Video Output: connection to S-Video [Y/C] inputs on external devices
- ⑮ Composite video output for connection to composite video inputs on external devices (composite cinch adaptor ⑬a is required)
- ⑯ SDI (Serial Digital Interface) Input: interface for external SDI video source
- ⑰ without function
- ⑱ SDI (Serial Digital Interface) Output: connection to external SDI video device
- ⑲ without function

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- ②0 DVI In: input to Smartscreen™ from AIDA DVD-M DVI out
- ②1 DVI Out: connection to Smartscreen™ DVI in
- ②2 Serial Data In: connection to Smartscreen™ from AIDA DVD-M Serial Monitor
- ②3 Microphone Input
- ②4 CD/DVD Open/Close Button
- ②5 CD/DVD Activity Indicator (flashes when writing or reading from/to disc)
- ②6 Power On Indicator
- ②7 Standby/On Switch
- ②8 Smartscreen™ Access Button (only on units with integral touch screen)
- ②9 Integrated Smartscreen™ Touch Screen Monitor
- ③0 Shipping lock for integrated Smartscreen™ touch screen monitor
Transportation: in the 'lock' position
Operation: in the 'unlock' position

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Important information for users of KARL STORZ equipment and instruments	..II
General warning	..II
Controls, displays, connectors, and their uses	..IV
Symbols employed	..VI
System description	..4
General information	..4
Features	..5
Warnings and cautions	..6
Definitions	..6
Safety Instructions	..6
Normal use	..9
User qualification	..9
Safety precautions at the site of installation	..9
Safety precautions when operating the equipment (electromagnetic compatibility)	..10
Unpacking	..11
Installation and operating instructions	..11
Basic equipment	..12
Connecting AIDA DVD-M	..13
Power supply	..13
Connecting AIDA DVD-M to the Karl Storz Image 1™ Camera Control Unit	..14
Connecting AIDA DVD-M to a Smartscreen™	..15
Connecting AIDA DVD-M to a printer	..15
Connecting AIDA DVD-M to a monitor	..15
Connecting AIDA DVD-M to a remote touch screen	..16
Connecting a microphone	..17
Connecting a mouse	..17
Connecting an external keyboard (optional)	..17
Preparing Smartscreen™ (20204520-1/-140 only)	..17
Operating de AIDA DVD-M	..18
Power up	..19
Power down	..19
Inserting discs, USB and information	..20
Disc insertion	..20
Disc care	..20

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USB Insertion	.20
Settings	.21
Record settings	.22
Print Settings	.25
System Settings	.26
Network Settings	.27
Advanced network settings	.28
Text Settings	.29
Info Settings	.29
Capture screen	.30
Inputting and editing of patient information	.31
Capturing Still Images	.32
Audio Recording During Still Capture	.33
Recording a video	.34
Audio capture during the video recording	.34
Access to still image summary or video summary and print preview	.35
Procedure Manager	.43
The Procedure Manager allows:	.43
File Recovery of an Incomplete Procedure within Procedure Manager	.43
Reviewing a File within Procedure Manager	.46
Copying a file to external media within Procedure Manager	.47
Batch File Copying to External Media within Procedure Manager	.48
Filtering from procedure lists	.49
Deleting Files from AIDA DVD-M Memory within Procedure Manager	.49
Viewing a Recorded Procedure on a Computer	.49
Help	.52
Cleaning and care of the unit	.53

Maintenance	.53
Maintenance	.54
Servicing and repair	.54
Disposal	.54
Limitation of liability	.55
Manufacturer's warranty	.55
Troubleshooting	.56

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Technical description	.56
Technical data	.58
Standard compliance (for 20204520-1/20204520-140)	.60
Directive compliance	.60
Technical documentation	.61
Software User Rights	.62
Optional Accessories	.63
Spare parts, recommended accessories	.63
Appendix	.64
Subsidiaries	.82

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Property rights

This product is protected in the USA by (at least one of the following) US Patent No(s) 5,788,688; 6,397,286; 6,484,221; 6,824,539.

System description

AIDA DVD-M is a digital video, still image and audio capture device intended for documentation of surgical procedures. The captured images or streaming video and associated audio may be archived to a CD, DVD, USB or network.

Note: AIDA DVD-M may only be used for documentation purposes. The images documented are not approved for diagnostics and/or therapy.

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Features

- User-friendly touch screen (remote or optional integral Smartscreen™) for control of functions
- Digital storage capability on CD-R, DVD+R or DVD-R, USB & Network
- Capture of images/video from the KARL STORZ programmable camera head buttons ACC1 and ACC2
- "Print during capture" for immediate still image prints
- Optional audio recording during video or still image capture
- Printed report listing patient information and thumbnail images of the captured still images
- For use with both NTSC and PAL video signals
- Video inputs: analog composite input, analog S-Video input, and digital SDI video input
- Video outputs: analog composite output, analog S-Video output, and digital SDI video output
In the Normal mode, all video inputs are "looped through" AIDA DVD-M directly to the monitor
- Audio inputs: Line level audio input, microphone level audio input
- Still image captures in DICOM format
- Multi-session recording on optical media

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Warnings and cautions

Warnings and cautions

Please read this manual and follow the instructions carefully. The words **Warning**, **Caution**, and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of the device. To make the words Warning and Caution stand out more clearly, they are accompanied by a pictogram.

Definitions

Warning: A Warning indicates that the personal safety of the patient or physician may be involved. Disregard of a Warning could result in injury to the patient or physician.

Caution: A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the unit.

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Warning: Read this instruction manual thoroughly and be familiar with its contents prior to using this equipment. Read the section on safety instructions carefully to avoid putting your patients, personnel, or yourself at risk.

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Warning: The instructions and interface specifications for medical devices and/or system components used in combination must be observed precisely.

Warning: Combinations of medical devices are only assured to be safe if

- they are identified as such in the respective instruction manuals or
- the intended purpose and interface specifications of the devices used in combination permit this (cp. IEC 60601-1-1).

Warning: Test this equipment prior to each surgical procedure to ensure that it functions correctly.

Warning: Risk of explosion if used in the presence of flammable anesthetics.

Warning: To avoid the risk of electrical shock, do not open the unit. Refer servicing to the manufacturer or to personnel authorized by the manufacturer. Removal of covers by nonauthorized personnel will void the equipment's warranty.

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Caution: Avoid allowing fluids entering the unit. Do not store liquids on or above the unit.

Note: Disposal of these system products, at the end of their useful life, must be in accordance with local regulations. This system contains a lithium battery (Li Mn O₂).

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Warning: Accessory equipment connected to the analog and digital interfaces must be certified according to the respective IEC standards (e.g. IEC 60950 for data processing equipment and IEC 60601-1 for medical equipment). Furthermore, all configurations shall comply with the system standard IEC 60601-1-1. Any person who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore responsible for ensuring that the system complies with the requirements of the system standard IEC 60601-1-1. If in doubt, consult the technical service department or your local representative.

Warning: To ensure safe operation, do not simultaneously touch the device output connectors and the patient.

Note: Before shipping, please disconnect **all** cables, including the Smartscreen™ serial data and DVI cables. Unit should always be transported in its original packaging. When available, the Smartscreen™ shipping lock should be used to secure the Smartscreen™ during shipping. To do this, hold in the Smartscreen™ and rotate the retention screw on the back panel until tight.

Note: Any damage to the unit resulting from incorrect operation is not covered by the manufacturer's warranty.

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Normal use

AIDA DVD-M is a digital image capture device intended for documentation of surgical procedures. The captured images or streaming video and associated audio may be archived to a CD or DVD. Use of AIDA DVD-M in other applications is not allowed for safety reasons.

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AIDA DVD-M may only be used with accessories, wearing parts, and disposable items which are designated by KARL STORZ as suitable for AIDA DVD-M or the safe use of which is proven. For safety reasons, do not perform unauthorized conversions or modifications to the device.

User qualification

AIDA DVD-M may only be used by physicians and medical assistants who have a corresponding specialized qualification and are instructed in the use of this equipment.

Safety precautions at the site of installation

The unit may only be used in medical rooms installed according to applicable national standards.

It is not intended for use in hazardous zones. This means, for example, that when using easily combustible and explosive inhalation anesthetics or mixtures thereof, AIDA DVD-M must not be operated inside the demarcated hazard zone. Examples of such substances are: anesthetic ether (diethyl ether, cyclopropane) as well as combustible, volatile skin cleansers and skin disinfectants which may create an explosive atmosphere (e.g. detergent ether, petroleum ether).

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AIDA DVD-M is equipped with a connector for attaching a ground line. It should be connected according to applicable national standards.

Caution: Degree of safety in the presence of flammable anesthetics: Equipment not suitable for use in the presence of flammable anesthetic mixture with air, with oxygen, or with nitrous oxide.

Safety precautions when operating the equipment (electromagnetic compatibility)

It is the user's responsibility to make sure the equipment is safe and operates properly before use.

CE marked equipment has been tested and found to comply with the EMC limits for the EN/IEC 60601-1-2 :2001 CISPR 11 Class A.

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Unpacking

Carefully unpack AIDA DVD-M and its accessories, avoiding extreme impacts to the device. Check for missing items and evidence of shipping damage. File any complaints with the manufacturer or supplier immediately. If possible, retain the original packing materials for later use; these can be used whenever the unit must be transported.

Note: For units with a SmartScreen™ shipping lock on the back panel, unscrew the shipping lock ③0 to release the screen.

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Basic equipment

20204501-140	AIDA DVD-M with Smartscreen™ integral touch screen, consisting of:
20204020-140	AIDA DVD-M Unit with integrated Smartscreen™
547 S	S-VHS (Y/C) video cable, 180 cm
536 MK	2 BNC connecting cables, 180 cm
200400 83	2 BNC cinch adaptors
20040088	USB recording extension cable, 75 cm
200400 84	Serial data cable, 20 cm
200400 85	DVI cable, 20 cm
400 A	Power cord (grounded)
400 B	Power cord (USA)
96206108D	Instruction manual, AIDA DVD-M

20204502-1	AIDA DVD-M without Smartscreen™ consisting of:
202040 20-1	AIDA DVD-M
547 S	S-VHS (Y/C) video cable, 180 cm
536 MK	BNC connecting cables, 180 cm
200400 83	2 BNC cinch adaptors
400 A	Power cord (grounded)
400 B	Power cord (USA)
96206108D	Instruction manual, AIDA DVD-M


Note: AIDA DVD-M recommended media includes CD-R, DVD+R and DVD-R. DVD+RW and DVD-RW media may be compatible, but are not supported or recommended for use with this product.

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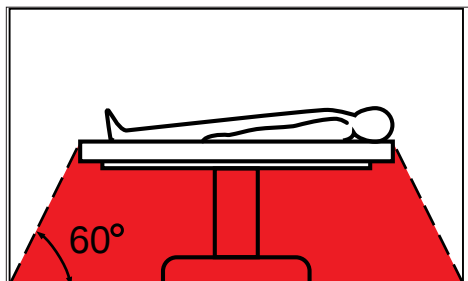Connecting AIDA DVD-M

Warning: AIDA DVD-M may be used only in medical facilities having electrical installations conforming to applicable national regulations.

Warning: AIDA DVD-M is not intended for use in hazardous zones. Do not operate AIDA DVD-M within demarcated hazard zones while explosive anesthetic gases are in use.

Note: AIDA DVD-M is equipped with a connector for attaching a grounding line. AIDA DVD-M grounding line should be installed by a qualified electrician.

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Power supply

Set AIDA DVD-M on a flat surface. Make sure there is sufficient distance on all sides to other equipment (especially radio frequency surgical equipment) and objects.

Before plugging in AIDA DVD-M, ensure that the voltage on the nameplate of 100-240V corresponds to the voltage of the local power line.

Connect grounding cord to the corresponding connector ②.

Connect power cord. Insert power cord into power cord receptacle ① as far as it will go.

Warning: Always use a hospital grade power cord with AIDA DVD-M. Insert the power cord only into electrical outlets located outside areas subject to explosion hazards.

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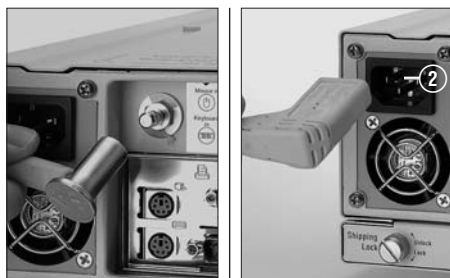
Note: It is advised that one output from the video source (CCU) be connected to the AIDA DVD-M input while a secondary output from the source is connected to a secondary input on the surgical monitor. This allows for switching to the secondary input of the monitor should the live image through AIDA DVD-M fail.

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Note: The arrangement or availability of connections can vary depending on the model used.

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Connecting AIDA DVD-M to the KARL STORZ IMAGE 1™ Camera Control Unit

For BNC cable connection: If using a composite video signal, you must use the enclosed composite cinch adaptor (13 a) to connect the BNC cable to the AIDA DVD-M on the back panel. This connection design was necessary due to space limitations on the back panel of the device.

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Connecting AIDA DVD-M to a monitor

If you wish to use an external monitor in addition to the Smartscreen, connect AIDA DVD-M ⑦ and the external monitor with the help of a VGA cable.

Note: Using a VGA output ⑦ requires the system to be restarted and reconfigured by a technician.

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Connecting AIDA DVD-M to a Smartscreen™

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Connect the supplied DVI cable to ⑳ and ㉑ (see p. III).

Connect the supplied RS232 cable to ⑥ and ㉒ (see p. III).

Connecting AIDA DVD-M to a printer

Note: Only the upper USB connection can be used for printer.

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Connecting AIDA DVD-M to a remote touch screen

Note: Only one touch screen may be used at one time. Multiple touch screen connections are not supported.

If you wish to use an external touch screen instead of the Smartscreen, connect the AIDA DVD- M connection ⑦ and the touch screen with the help of a VGA cable. Also insert an RS232 cable between the AIDA DVD-M ⑥ and the touch screen.

Note: Using a VGA output ⑦ may require the system to be restarted and reconfigured by a technician.

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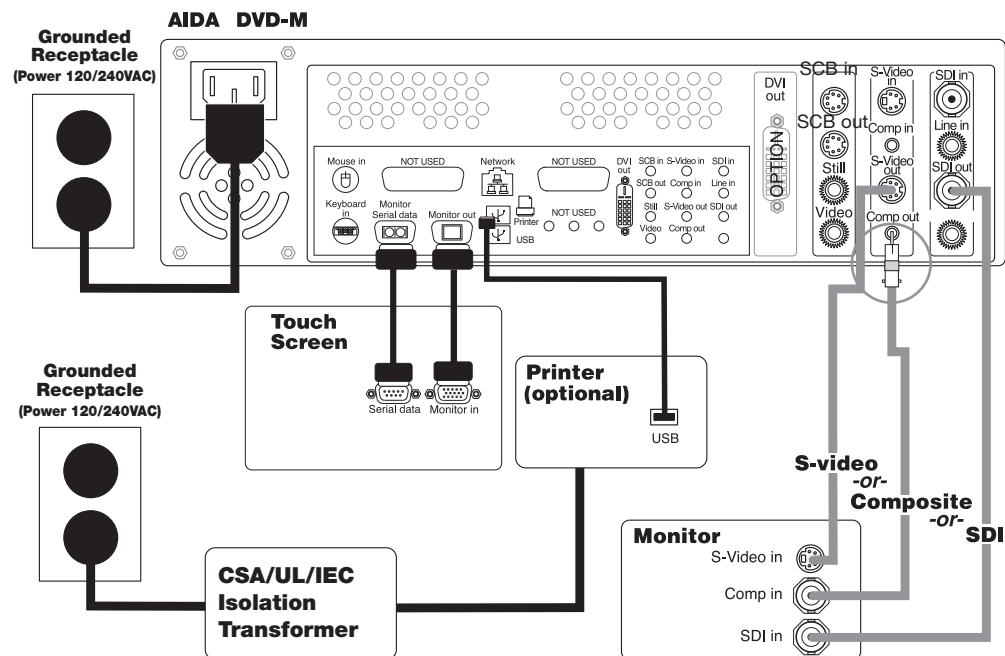
Connecting a microphone

If audio recording is desired, attach the microphone plug into the microphone input ②③ on AIDA DVD-M front panel.

Note: The microphone is optional. For ordering information, contact your KARL STORZ sales representative.

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Connecting a mouse

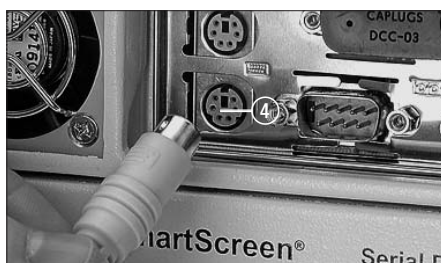
If a computer mouse will be used to control the AIDA DVD-M functions, plug the mouse into the 'mouse in' input ③ on the AIDA DVD-M back panel.

Note: The mouse must be plugged in before turning on the AIDA DVD-M in order to function.

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Connecting an external keyboard (optional)

For input of data an external keyboard can be used (see accessories). Plug the keyboard into the 'keyboard in' input ④ on the AIDA DVD-M back panel.

Note: The keyboard must be plugged in before turning on the AIDA DVD-M in order to function.

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Preparing Smartscreen™ (20204520-1/-140 only)

Before you use the Smartscreen™ touch screen for the first time, check that the shipping lock has been released. If necessary, turn the shipping lock ③① on the back of the unit to the 'Unlock' position (see arrow).

If your AIDA DVD-M includes the Smartscreen™ integral touch screen, press the touch screen access button ②⑧ and pull out the Smartscreen™. Move the bottom of the touch screen up or down for the desired viewing angle.

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Operating AIDA DVD-M

The touch screen (remote or Smartscreen™) is used to access all functions of AIDA DVD-M, including:

- Setting system preferences
- Capturing still images, streaming video, and audio
- Reviewing, printing, and saving captured files

Caution: To select a function, touch the screen with your finger. (Do not use a pencil or pen to touch the screen, as this may cause damage).

If a computer mouse will be used to control the AIDA DVD-M, an on-screen pointer is available to point to the desired function. Press either mouse button to select.

The following symbols are used to navigate through the screens:

Accepts input and moves to the appropriate screen

Cancels new input and returns to previous screen

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Power up

Warning: Test AIDA DVD-M before each procedure. Ensure that the proper video image appears on all video monitors before beginning each procedure.

Press the Standby 27 button on the front panel. If a printer will be used during the session, turn printer on. Do not turn the printer off until the session is completed.

Note: When powering up for the first time, a series of screens will appear asking the user to press specific areas of the screen. With this response, the system will automatically calibrate the touch screen in use. This calibration will only occur when AIDA DVD-M is turned on for the first time, when new software is installed, or when a different touch screen is used. Next, and with subsequent power ups, a welcome screen will appear.

You can choose seven languages (English, French, Portuguese, Italian, German, Russian and Swedish). The language can be selected under 'Settings' / System / Language. You can select the language you require there. For further information, please refer to the section 'Settings' on p. 21.

Power down

To switch off the unit, always press the Standby/Power switch at the front and hold down for a few seconds.

Caution: Switching off by any other method, such as using the power switch on the equipment trolley, may result in file corruption.

Note: The AIDA DVD-M should be powered down at the end of each working day.

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Inserting discs, USB and information

Disc insertion

A disc or a USB may be inserted into AIDA DVD-M at any time during a procedure. AIDA DVD-M recommended media include CD-R, DVD+R and DVD-R. DVD+RW and DVD-RW media may be compatible, but are not supported or recommended for use with this product.

Note: If there is a plug covering the port labeled USB on the back of the device, pull the plug out of the port (The purpose of this plug is to discourage users from inserting a USB printer in that port).

Touch ▲ 24 to open the disc drawer. Insert a blank disc as desired and touch ▲ 24 again to close the disc drawer. If the inserted disc is an incompatible format, a message will appear with further information.

Disc care

Use only clean, dry discs. Do not touch, soil, or scratch the recording surface (underside). If necessary, clean recording side with a soft, dry cloth. Store discs in a disc holder or sleeve for protection.

USB Insertion

A USB device may be directly inserted into the lower labeled USB input on the back of the AIDA DVD-M at any time during a procedure. The included USB extension cable may also be used to allow for insertion and removal of USB devices from the front of AIDA DVD-M.

Note: A USB device always has to be connected to the lower USB connector on the rear side of the AIDA DVD-M.

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Settings

Before beginning a capture session, check the settings to ensure that AIDA DVD-M is set to desired preferences.

To access the "Settings" options, touch  on the upper right of the AIDA DVD-M Capture or Review/Print screens. Upon opening the Settings options, the screen of the last opened Settings tab will appear. Touch the desired tab to access the various Settings options.

Touch  located on the upper right of screen to exit the Settings options from any screen. When prompted, touch either  to save the input information and return to the previous screen or  to return to the previous screen without saving changes.

The settings are explained as follows:

- Record Settings (record)
- Print Settings (print)
- System Settings (system)
- Network Settings (network)
- Advanced Network Settings (network / )
- Text Settings (text)
- System information (info)

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Record settings

Touch the 'Record' tab to access the 'Record' screen displayed on the left. You can select the following settings:

- **Still Image Resolution:** Select resolution (640 X 480, 800 X 600, 1024 X 768) for storing still images.

- **Still Image Format:** Select format for storing still images.

JPEG: .jpeg files are uncompressed, high quality, large files used for publishing.

TIFF: .tiff files are highly compressed, good quality, small files used in websites

Bitmap: .bmp (bitmap) files are uncompressed, high quality files compatible with most applications.

- **Video Format:** Select format for storing video. Currently, MPEG-2 is the only available format.

- **Video Quality:** Select quality of stored video: (High or Medium quality).

Note: The higher the quality, the more disc space is used and more time is necessary to retrieve images/videos. (i.e. On an empty disc, 'Medium quality' allows approximately 20 minutes of CD recording or two hours of DVD recording; 'High quality' allows approximately 14 minutes of CD recording or 1.5 hours of DVD recording.).

- **Video Source:** Select camera video input connection type. The selected video input will be displayed on the surgical monitor.

- **HD Video Standard:** If connected to a Matrox HD Box, select the appropriate option according to the local standard (1080p60 or 1080p50).

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Access Settings

Note: Automatic (Auto) allows the system to detect and select the attached video input connection. If more than one video input is connected and video source option is set to Auto, the system chooses the video connection to display in the following order: SDI, S-video, composite.

• **Enable Audio:** Select YES or NO to enable audio recording during video capturing. If NO is selected, the microphone icon  will no longer appear on the capture screen.

Note: Some third party editing software programs may not perform well if audio files are embedded within video streams. Please consider this prior to recording audio with videos intended to be edited at a later time.

• **Capture Audio with Stills:** Select Yes or NO for audio recording during still image capturing. If NO is selected, audio recording will be unavailable during capture. If YES is selected, five seconds of audio will be captured with each still. When a still is captured, an on-screen message 'still/audio' will appear and will be replaced with the message 'audio' during the five seconds of recording. When audio for stills is enabled, the microphone icon  will appear on the Capture screen. When a still is taken, the microphone icon will turn green to confirm audio recording.

Note: Yes must be selected for "Enable Audio" in order to select Yes to capture audio with stills.

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• **Enable DICOM for Stills:** Select YES or NO to enable DICOM formatted still images. If YES is selected, all still images in an ended procedure will be saved as both the selected "Still Image Format" (.tiff, .jpg, or .bmp) and a .dcm file.

• **On-Screen Indicator:** When recording a still image (and Print During Capture option is selected – See Print Settings Options section), an on-screen print count indicator will appear. For example if four prints per page are selected, when the first still image is taken, the on-screen displayed message will be "1 / 4". This count will remain on the screen if the On-Screen Indicator 'Yes' option has been selected. In addition, when recording a video, an on-screen message 'rec video' will appear on the monitor at the start of recording. After a few seconds, the message will be replaced by a small steady green square icon to indicate that the video is being recorded. At end of recording, an on-screen message 'rec stop' will appear on-screen. The green recording icon can be turned On or Off from being displayed on the viewing monitor by selecting Yes or No. The system default is set to On which will display the indicators for stills and videos.

Note: Messages generated by AIDA DVD-M could interfere with SCB messages that are displayed in the same location on the surgical monitor.

• **Still Enhancement:** Select 'Yes' or 'No' to increase the apparent detail in the captured still images. This is strictly a preference option.

• **HD Still Watermark** If connected to a Matrox HD Box, and capturing still images with Image 1 HD Camera head, selecting 'Yes' will incorporate an HD Watermark into the captured still image file and the printed still image.

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Print Settings

Note: The Print settings can be changed during a session (with the exception of 'Select your printer').

Touch The Print tab to access the Print options screen at left. The following settings can be selected:

- **Select Your Printer:** Use the up or down arrow to scroll through the list of printers and select the desired option.

Note: The system's software will populate the printer selection field based upon what printers are installed on system. In addition, once a printer has been selected and new settings have been applied, a small printer icon will become visible in the upper right corner of the user interface screen.

Note: When using the print option please make sure that the printer is connected to the upper USB port.

Note: The system's software will populate the printer selection field based upon what printers are installed on system.

- **Page Layout:** Select the page layout for printed images. The three layout choices include: No Text, Text, or Advanced.

- **Images per Page:** Select the number of images printing on one page (1, 2, 4, or 8).

- **Copies:** Select the number of copies of each page desired.

- **Print during Capture:** Select the ability to print still images immediately after they are captured, and the selected page layout is full.

Note: The feedback display on the monitor changes irrespectively of whether or not 'Print during capture' is activated.

- **Print Report:** Select the ability to print a report which includes patient information, a list of captured video files, and thumbnail prints of each still image. If 'Yes' is selected, this report will automatically print at the end of each procedure.

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System Settings

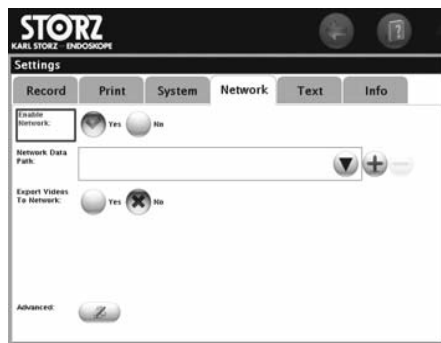
Touch the System tab to access the System options screen at left. You can select the following settings:

- **Language:** Select the desired language.
- **Date format:** Select dd/mm/yyyy, mm/dd/yyyy, or yyyy-mm-dd format.
- **Time Format:** Select 24 hour or 12 hour format.
- **Hour, Minutes, Second:** Select the desired time.
- **Date:** Use the calendar to select the desired date.
- **Calibrate Touch screen:** Touch . On the next screen, press each green circle that that appears to calibrate the AIDA DVD-M touch screen. The touchscreen may be recalibrated at any time during a procedure.

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Network Settings

Warning: This device should only be connected up to a medical hospital network system that complies with the national regulations for health facilities or an equivalent local standard.

Note: The AIDA DVD-M provides basic networking capability to send information to a configured target address. This network support is for one direction only – sending information out of the AIDA DVD-M to a configured target address.

Touch the 'Network' tab to access the screen displayed on the left. You can select the following settings:

- **Activate the network memory:** Activate or deactivate the memory using Yes or No.
- **Network path:** Select the network path: Touch ▼ to choose from a list of previously entered network destinations. Enter user name and password when asked.

Touch + to add a new network address to the saved list.
Touch - to delete a present network path from the list.

- **Transferring the videos to the network path:** Activate or deactivate the video export using Yes or No.

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Advanced network settings

Note: It is recommended that the Advanced Network Settings for AIDA DVD-M be configured by the hospitals IT department.

To access the 'Advanced' screen, open the advanced network settings with . You can select the following settings:

• Keeping user name and password:

Activate one of the following possibilities:

 until the system is shut down.

 only for this procedure.

 always

Touch the desired field and use the keypad to enter the following information:

• **Connection type:** Select the desired network connection.

• **IP address:** Enter the desired IP address.

• **Subnet mask:** Enter the desired subnet mask.

• **Standard gateway:** Enter the desired standard gateway.

• **Preferred DNS server:** Enter the preferred DNS server.

• **Alternative DNS server:** Enter the preferred alternative DNS server.

• **Set user name and password:** Touch  to enter the user name and password.

• **Delete user name and password:** To delete password.

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Text Settings

Touch the 'Text' tab to access the 'Text' screen displayed on the left.

The text settings make it possible to add (or delete) operator or procedure names in stored lists. The name of the hospital can also be entered.

Touch to save the input information.

Touch to delete the selected name from the saved lists.

Note: The saved Surgeon and Procedure lists are available when entering patient information.

Hospital Name: Use keyboard to input hospital name. Once saved the hospital name will appear on the procedure report and any printed still images.

Info Settings

Touch the Info tab to access the Info screen. This screen displays the following information about this specific AIDA DVD-M device.

- Software version
- Serial number
- Chassis material number
- SmartScreen™ material number
- Video firmware version
- DVD version
- BIOS version
- Main board version
- Drive version
- Memory

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Capture screen

After the start screen, the capture screen 'Capture' on the right appears. Here, you are inside the first procedure.

Note: When the camera is connected and the settings are correct, you can see the live picture as a preview in the capture screen. The following steps can be taken during a running operation.

This screen allows:

1. Inputting and editing of patient information
2. Capturing still images, video and audio
3. Access to still image summary or video summary and print preview
4. Access to Procedure Manager
5. Access to Help Files
6. Opening of configuration settings

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Installation and operating instructions

1. Inputting and editing patient information

Inputting and editing of patient information

After powering up the AIDA DVD-M, the Capture screen will appear (see picture on p. 30).

Note: A procedure may be started with or without first entering patient information. However, certain required patient information (fields highlighted in yellow on patient information screen) must be entered prior to completion of the procedure. The system will prompt the user to enter required patient information fields at the end of a procedure if this task has not yet been completed.

To enter patient information, touch the hyperlink „Patient data“ in blue at the top left of the Capture screen. A keyboard will appear to allow input of the patient information. To input patient information into each field touch the desired field.

Type in the desired information, or use the up or down arrow to scroll through a list of saved names in the „Surgeon“ and „Procedure“ fields and select the desired entry. Surgeon and Procedure names may be added to the saved lists in the Text option in Settings (see „Settings“ page 21).

Keyboard

- **Backspace:** Erases back one space; hold down backspace key to continue erasing.
- **Tab:** Moves cursor to the next field.
-  : Moves the cursor in the direction of the arrow to allow inserting of text
- : Saves the input information and returns to the previous screen
- : Returns to the previous screen without saving changes.

Note: The Year field requires four numbers. The format of the „Date Of Treatment“ and „Date of Birth“ fields can be set in the System option in Settings (see „Settings“ section of the manual for more information, page 25).

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2. Capturing still images, video and audio

Capturing Still Images

Via touch screen:

When the desired image is on-screen, touch the still image icon  to capture a still image. The icon will highlight during capture, and the captured image will freeze momentarily on the touch screen and on the main surgical monitor to indicate a still image has been captured. On the left side of the screen the system indicates how many still images have been captured.

Via camera head buttons:

To capture an image using the camera head buttons, an Accessory cable from the camera ACC 1 output must be connected to the still input on AIDA DVD-M (see p.14). In addition, a camera head button must be programmed to capture still images. Once the desired image is on screen, press the programmed camera head button to capture the image. The captured image will freeze momentarily, a beep will sound, and the image will be displayed on the main monitor as described above in the “Via touch screen” section.

Note: The optional two pedal footswitch can also be used to capture still images and video when wired to the “still” and “video” inputs on AIDA DVD-M.

Print during capture feature:

If AIDA DVD-M is set to “Print during Capture”, captured images will be printed immediately once a page is full (“full” is based on selection of 1, 2, 4, or 8 images per page set in the Print options in Settings). Any printed pages which are not full when capturing is completed (and therefore have not printed) will print automatically upon ending the procedure.

Note: Still images can be captured while recording the video.

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Audio Recording During Still Capture

To record audio during still capture, a microphone must be connected to the microphone input on AIDA DVD-M. “Capture audio with stills” must also be selected in the Record options in Settings. If capture audio with stills is enabled, five seconds of audio will be captured with each still. When a still is captured, an on-screen message ‘still/audio’ will appear and will be replaced with the message ‘audio’ during the five seconds of recording. When audio for stills is enabled, the microphone icon  will appear on the Capture screen. When a still is taken, the microphone icon will turn green for five seconds to confirm audio recording.

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Recording a video

Using Touchscreen:

Touch .  turns green. Capturing begins. On the left of the screen, the number of captured videos is displayed under Videos (including the latest one) and the total recording time is shown in black. The time of the current recording is shown in red.

To stop the video capture, touch  again.  turns grey again.

Using the camera head buttons:

To capture images using the camera head buttons, an accessory cable must connect Output 1 or ACC 2 of the camera to the Video Input on the AIDA™ DVD-M (see p.14). A camera head button must also be programmed for the image capture.

Note: The optionally available footswitch with two pedals can also be used for the capture of still images and videos, so long as it is connected to the inputs 'still image' and 'video' on AIDA DVD-M.

Audio capture during the video recording

Note: If 'Activate audio capture' has been activated, the symbol  is visible. You now have the possibility to activate the audio capture.

To record the tone, connect a microphone to the microphone input on AIDA DVD-M. Activate 'Activate audio' under 'Settings'/record. The microphone symbol becomes active once the video capture starts. To start the audio capture, touch . The symbol turns green. To stop the audio capture, touch  again. The symbol turns grey again.

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3. Access to Review/Print screen

Access to still image summary or video summary and print preview

The screen Still image summary/video summary/print preview allows:

- reviewing still image and video files
- capturing stills from previously recorded video
- renaming or editing of files
- printing still images
- deleting still image and video files
- ending and save a procedure
- returning to the Capture screen

Reviewing still images and video

Still images: To access the Review/Print screen, touch any still image or video thumbnail. If there are more files than will fit in the Still/Video review area, touch ▼ or ▲ arrow to scroll through additional thumbnails. Touch the desired thumbnail image to view the image on the larger viewing screen on the touch screen.

Video: Touch the desired thumbnail image to view captured video on the touch screen. Videos that are longer than 12 minutes will be broken into multiple 12 minute files.

Touch ► to play the video. Touch or drag the indicator on the timer bar to move the video forward or back.

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Capturing Stills from Video

Additional still images can be captured from previously recorded video stream during video review using the AIDA DVD-M touch screen.

In the Review screen of the video files, touch  to capture stills from previously recorded video. Stills may be captured whether the video is paused first or whether the still is captured while the video is actively playing back previously recorded information. These captured stills will be filed in the Still/Video area and may be printed as described above in the “Printing still images” section.

Naming or renaming captured files

To rename a captured file, touch the thumbnail image then touch the “File Name” box. A keyboard will appear. To replace or add to the displayed file name, enter the desired information in the highlighted field. You may also input or edit file comments in this screen. When complete, touch  to save changes and return to the Review/ Print screen, or touch  to return to the Review/ Print screen without saving changes. New file names will display in the “File Name” box.

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Printing still images

You can print images during the capture. You can find information on the 'Print during Capture' option in the sections 'Capture still images' (p. 32) and 'Settings' (p. 21).

If the 'Print during capture' option was not activated at 'Settings', images may be printed at any time before or after a procedure.

Touch the desired thumbnail image to display the image on the 'Still Review' screen. Then touch  to place the image in the print queue. To send further images to the print queue, repeat these steps.

According to the settings of the page layout and selection of 1, 2, 4 or 8 images per page, the images are added to the print queue via 'Settings'/'Print'.

Touch  to print all the images in the print queue.

To end the procedure, touch .

Print preview: Touch  in the window 'Review/Print', to open the print preview. The right page shows a preview of the images in the print queue. With  or  you can navigate your way through the print queue pages.

Deleting an image from the print queue:

Highlight the image to be deleted by touching the thumbnail images in the print preview. Then touch .

To add a new image into the empty space in the print preview, highlight the empty space by touching and then touch the required thumbnail image. The selected thumbnail image will be placed in the empty highlighted location in the print queue.

Note: An image deleted from the print queue will remain in the procedure files.

Note: In order to leave the preview, you can at any time touch the  symbol on the lower far left of the screen.

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3. Access to Review/Print screen

Deleting still images and video files

In the 'Still image summary or video summary' window, touch the thumbnail image to be deleted. Then touch the  symbol. Confirm your wish to delete using the symbol . Touch  to cancel the delete process.

Ending and saving procedure

After selecting the desired images and/or videos to save, touch  to move to next screen.

A message box with the following options will appear:

- Touch  to end the procedure.
- Touch  to continue the procedure.

Note: An ended procedure may not be changed. Ended procedures are temporarily saved on the AIDA™ DVD-M hard drive for copying to external media or a computer network (if configured in the Network option in Settings).

Note: If any of the mandatory patient information fields are blank, a keyboard will appear at this point to allow input of the required patient information.

A procedure may not be ended without completion of all the required patient information (fields in yellow). Once all the required patient information has been entered, touch  to proceed to the next screen.

After successfully ending a procedure, a message box with the following options will appear:

1. copy procedure to external media
2. start a new procedure
3. shut down AIDA DVD-M

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3. Access to Review/Print screen

1. Copying a Procedure to DVD or CD

1. Touch . A message box with the following options will appear:

- copy procedure to CD/DVD
- copy procedure to USB
- copy procedure to a network

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Note: Multiple procedures may be copied to a single disc. (depending upon available space). However, discs that have been closed will not allow the copying of additional procedures.

a) Copy procedure to CD/DVD

- Touch .
- Select DVD/CD as the copy destination.
- When prompted, insert disc. Touch  to open and close the disc drive tray.
- AIDA DVD-M will check the inserted disc to ensure it is valid and that the procedure will fit on the disc.
- When prompted, touch right arrow to begin copying.
- When copying is successfully completed, the “Next Step?” screen will again appear.

Caution: To avoid information loss during disc writing:

- do not eject the disc
- do not turn the AIDA DVD-M power off.

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b) Copying a procedure to USB

1. Touch .
2. Select USB as the copy destination.
3. AIDA DVD-M will prompt the user to insert a valid USB device (if not already inserted) and check the device to ensure it is valid and that the procedure will fit on the device.
5. When prompted, touch right arrow to begin copying.
6. When copying is successfully completed, the "Next Step?" screen will again appear.

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c) Copying a procedure to Network

1. Touch .
2. Select Network as the copy destination.
3. AIDA DVD-M will check the network configuration (as set in Network options in Settings).
4. When prompted, touch right arrow to begin copying.
5. When copying is successfully completed, the "Next Step?" screen will appear.

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2. Starting a New Procedure

Touch to start new procedure. A new procedure may be started without copying the previous procedure to external media.

Caution: However, the AIDA DVD-M is not intended to be a permanent storage device. Procedures stored in the AIDA™ DVD-M memory are deleted automatically (with no notification to the user) as new procedures are captured (the oldest recorded procedure is replaced by the new procedure). To avoid loss of saved procedures, it is strongly recommended that each procedure be copied to DVD/CD, USB, or the network at the end of each procedure or capture session.

For further information, see “File recovery from memory” in the Procedure Manager section on p. 43.

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Procedure Manager

To access the Procedure Manager screen, touch  on the upper right of the AIDA DVD-M Capture or Review/Print screens.

Select one of the following information sources (AIDA DVD-M hard disk, CD/DVD, USB or network) using the buttons at the top of the screen.

The Procedure Manager allows:

- incomplete file/procedure recovery from memory
- file/procedure review from device memory, DVD/CD, USB, or the network
- access to the Reference screen
- file/procedure copying from device memory *
- batch file/procedure copying to external media from device memory *
- filtering procedure list
- file/procedure deleting from device memory *

***Note:** Files retrieved from CD, DVD, USB, or the network for viewing within the Procedure Manager can only be reviewed and/or printed. These files cannot be copied to any external media or deleted.

File Recovery of an Incomplete Procedure within Procedure Manager

Caution: The maximum number possible of recent procedures, based upon file sizes, are stored on the AIDA DVD-M memory for future recovery. Stored procedures are deleted automatically (with no notification to the user) as new procedures are captured (the oldest recorded procedure is deleted/replaced by the new procedure).

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4. Opening Procedure Manager

Any procedure(s) not properly ended (i.e. procedure was ended without entering required patient information) is marked „**incomplete**“ in red and stored in the Procedure Manager. Incomplete procedures can result from improper finalization of a procedure, improper power shut off, or power failure. Patient information in incomplete procedures may be accessed and edited. However, still images and videos may not be added or removed from the incomplete procedure.

To recover an 'incomplete' file/procedure from memory:

1. Touch  to open the Procedure Manager screen.
2. Touch  to display the previously recorded procedures. The content of the AIDA DVD-M memory is shown in table form.
3. Use the arrow icons or the left-right and up-down slide bars to scroll through the list of available procedures.

Note: The list may also be sorted in ascending or descending order by touching the desired heading (Date, Surgeon, etc.). An up arrow or down arrow will display to note ascending or descending sorting.

Touch the selected heading again to toggle between ascending or descending order.

4. Touch any field noted in red as „incomplete“. The selected incomplete procedure will become highlighted in blue and will display small thumbnail images of the still images and video files in the procedure.

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4. Opening Procedure Manager

5. Touch  in the highlighted procedure to view the entire incomplete procedure in the Reference screen. The Reference screen functions in the same manner as the Review/Print screen. Please see tp. 35 for more information.
6. Touch the 'Current' and 'Reference' tabs to toggle between the live Capture screen (live image) and the previously recorded procedure. Touching the  may also be used to return to the live Capture screen.
7. Touch 'TOUCH HERE TO ENTER PATIENT DATA' in blue at the top of the Reference tab. A keyboard will appear to allow input of the patient information as described in the 'Entering patient information' section on p. 31.
8. Once all the required patient information has been entered, touch  to proceed to return to the Reference screen.
9. Touch right arrow icon to end the procedure. See the 'Ending a Procedure' on p. 39 for more information.
10. To close the Reference screen without properly ending the incomplete procedure, touch .

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Reviewing a File within Procedure Manager

Previously recorded procedures stored in the AIDA DVD-M memory or on previously recorded CD, DVD, USB device, or stored to a hospital's network may be reviewed on the AIDA DVD-M touch screen through the Procedure Manager. In addition, previously recorded procedures may be viewed during a current procedure, but they may not be edited. Still images from previously recorded procedures may be printed during the review process. Stills and videos from previously recorded sessions cannot be added to any current procedures.

Reviewing a File within Procedure Manager:

1. Touch  to open the Procedure Manager screen.
2. Touch the icon of the desired information source (AIDA DVD-M memory, DVD/CD, USB, Network) to display the list of AIDA DVD-M procedures available for review.

Note: If DVD/CD or USB is selected, a disc or USB device must already be inserted into AIDA DVD-M.

Further procedure is analogous to 'To recover an 'incomplete' file/procedure from memory' on p. 44.

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Copying a file to external media with in Procedure Manager

Note: The only files that can be **copied** to external media are files stored in the AIDA DVD-M memory.

Files being reviewed from CD, DVD, USB, or the network within Procedure Manager cannot be copied to any external media. In addition, these files cannot be deleted.

Instructions for batch file copying

1. Touch  to open the Procedure Manager screen.
2. Touch the  to display the list of AIDA DVD-M procedures available for copying.
3. Touch the desired procedure. It will become highlighted in blue and will display small thumbnail images of the still images and video files in the procedure.
4. Touch  to copy.
5. On the next screen, select the desired copy destination as described in the “Ending and Save a Procedure” section p. 39.

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Batch File Copying to External Media within Procedure Manager

Note: The only files that can be batched to copy to external media from within the Procedure Manager are files stored in the AIDA DVD-M memory. Files being reviewed from CD, DVD, USB, or the network within Procedure Manager cannot be copied to any external media. In addition, these files cannot be deleted.

Files from the device memory may be copied in batches (two or more at a time) to the same media. This function may be useful in copying multiple procedures from a single capture session instead of copying each procedure separately.

Instructions for batch file copying:

1. Touch  to open the Procedure Manager screen.
2. Touch the  to display the list of procedures available for copying.
3. Touch the desired procedure. It will become highlighted in blue and will display small thumbnail images of the still images and video files in the procedure.
4. Touch  to select the procedure for batch copying. The procedure will move down into the lower 'batch' area on the screen. Touch  to move the procedure from the batch area back to the stored procedures area.
5. Repeat step 4 until all the desired procedures are in the batch area.
6. Touch  to copy procedures.
7. On the next screen, select the desired copy destination as described in the 'Ending and saving procedures' section p. 39.

Note: Touch  to move all files from the batch area to the Procedure Manager area.

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Filtering from procedure lists

Touch to filter or find procedures from the selected information source in Procedure Manager. A keyboard will appear. Touch the field corresponding to desired heading (Date, Surgeon, etc.) and type the desired information. The upper screen will find and display procedures that fit the entered filtering criteria. Touch again to exit the filtering function.

Deleting Files from AIDA DVD-M Memory within Procedure Manager

Touch first and afterwards to permanently delete the selected (highlighted in blue) procedure from the AIDA DVD-M memory.

Note: Saved procedures from a previously recorded DVD/CD, USB, or computer network cannot be deleted via the AIDA DVD-M Procedure Manager.

Viewing a Recorded Procedure on a Computer

To view a saved procedure on a computer, insert the recorded AIDA DVD-M media (CD, DVD, USB) into computer's appropriate drive.

An index screen will appear to allow access to recorded files with the still images and videos.

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4. Opening Procedure Manager

To view multiple recorded DVD sessions on a Macintosh with OSX, perform these steps:

1. Insert recorded DVD disc into MAC.
2. Click on Finder icon.
3. Click on AIDA DVD-M icon.
4. Double Click on OSX.
5. Double Click on Multisession.zip
6. The zip file moves to the Desktop (Multi Session Folder).
7. Open the multisession file from the Desktop.
8. Double Click on Open Disc folder.
9. An administrative window will open prompting the entry of the administrative password for the MAC being used. Enter the administrative password for the MAC and hit <Return>.

Once complete, the following message will appear:

Media re-mount succeeded

Logout

Process completed

10. Close the "Completed Command" dialogue box.
11. Click on any other Finder icon to move to next step.
12. Click on AIDA DVD-M again.
13. Click on Index.html to access all recorded sessions on disc.

Note: To import a video into iMovie, click and drag the video into the iMovie workspace.

Note: To view a DVD+R/-R procedure on a computer, the computer must have a DVD drive which is compatible with the DVD+R/-R format.

Note: AIDA DVD-M records mpeg2 videos. This codec must therefore be installed.

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Determine the format of your DVD drive:

Windows 2000 / XP / Professional:

1. Double click on "My Computer" icon on the Desktop.
2. Right click on the disc drive icon.
3. Select "Properties". Select "Hardware" tab.
4. Record Make and Model of DVD drive.
5. Go to website 'www.DVDplusRW.org' to check compatibility with DVD+R/-R.
6. If your drive is not compatible with a DVD+R/-R disc or is not listed, please contact KARL STORZ Technical Support at +49-7461-7080 or your local KARL STORZ representative.

Macintosh OS X:

1. Click the "Apple" icon on the Desktop.
2. Click "About this Mac".
3. Select "More Info". Select "Hardware".
4. Select "Disc Burning".
5. Record Make and Model of DVD drive.
6. Go to website 'www.DVDplusRW.org' to check compatibility with DVD+R/-R.
7. If your drive is not compatible with a DVD+R/-R disc or is not listed, please contact KARL STORZ Technical Support at 800-421-0837.

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Help

The 'Help' section provides setup and operating instructions. To access this section, touch  located at the top right of touch screen.

Touch  to return to the previous screen.

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Cleaning and care of the unit

Warning: Always pull out power plug before cleaning!

Caution: To clean the system, wipe with a damp cloth or sponge.

The exterior of the AIDA DVD-M can be cleaned with a conventional surface disinfectant.

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Handle the system with care, avoiding extreme impacts to device. When transporting, use the original shipping box to prevent damage to the device.

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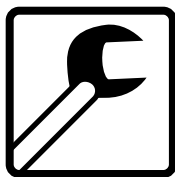
Store unit out of direct sunlight and excessive heat.

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Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, thus enhancing the instrument's reliability and extending its useful operating life. Maintenance services can be obtained from your local representative or from the manufacturer.

Regardless of the accident prevention regulations or testing intervals for medical instruments prescribed in different countries, we recommend a functional or safety test of the unit at least once a year. For detailed instructions please refer to the version of the Service Manual currently valid.

Warning: To ensure safe operation of AIDA DVD-M, you should perform the following procedures every 12 months:

1. Check leakage current from chassis $< 100 \mu\text{A}$, single fault condition.
2. Check ground impedance $< 0.1 \text{ ohms}$.
3. Check power consumption $\leq$ rated output.

Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original KARL STORZ parts only.

Disposal

The AIDA DVD-M has been marked in accordance with the European Directive 2002/96/EC on waste electrical and electronic equipment (WEEE). At the end of its useful operating life, dispose of the unit as electronic scrap. Please ask either KARL STORZ GmbH & Co. KG, a KARL STORZ subsidiary or your specialist dealer for information on your local collection point. Within the scope of application of this Directive, KARL STORZ GmbH & Co. KG is responsible for the proper disposal of this unit.

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Limitation of liability

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- all electrical installations at the location of use meet applicable national and local electrical codes, and
- the unit has been used in accordance with its operating instructions at all times.

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Manufacturer's warranty

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In so doing we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Please fill out the attached warranty card on the last page and return it as soon as possible to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Germany

Note: The unit must always be sent to the following address (also during the warranty period; where applicable, with warranty card):

KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Str. 34
78532 TUTTLINGEN/GERMANY

Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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Troubleshooting

Symptom

AIDA DVD-M 'locks up'; touch screen keys/camera head buttons will not respond and touch screen freezes.

Possible causes

The AIDA DVD-M operates similar to a computer. Just as with any computer, there may be an occasional 'lock up' of the system, due to no fault of the user.

Remedy

Turn the AIDA DVD-M OFF and ON to re-start the system (press button **⏻** twice).

To resume a session in-progress, access the AIDA DVD-M's Procedure Manager and select the session to recover.

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Note: Any prints remaining in the queue will print once the user ends the Procedure Manager.

Symptom

Still images cannot be captured.
Possible causes
Printer is not connected.
Printer was turned OFF or disconnected during a session.

Remedy

Connect/turn ON the printer.
In order to establish the connection to the AIDA DVD-M, turn the AIDA DVD-M OFF and ON again.
To resume a session in-progress, access the AIDA DVD-M's Recovery Procedure and select the session to recover.

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USB is not recognized by AIDA DVD-M.

Possible causes

USB device not completely connected or connected to the wrong USB-Port.
USB is invalid.

Remedy

Check USB and/or USB extension cable connection.
Insert a different USB device.
Ensure USB is connected to correct, labeled USB input.
Ensure one USB memory device is attached at one time.

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(for 20204520-1/20204520-140)

According to IEC 60601-1, UL 60601.1, CSA C22.2 601.1:

- Type of protection against electric shocks:
Protection Class I
- Degree of protection against electric shocks:
Applied parts of type BF 

IEC 60601-1-2 :2001:

Please read the Electromagnetic Compatibility Information in the appendix (p. 64ff).

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Directive compliance

(for 20204520-1/20204520-140)

According to Medical Device Directive (MDD):

This medical product belongs to Class I.

This medical product bears the CE mark in accordance with MDD 93/42/EEC.

A code number after the CE mark indicates the responsible notified body.

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согласно Medical Device Directive (MDD):

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Technical documentation

On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified user personnel duly authorized by the manufacturer for their use in repairing those components of the unit that have been designated by their respective manufacturers as repairable.

Supply of such technical documentation relating to the unit shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the unit.

Explicitly exempted herefrom are those maintenance and repair operations described in this manual.

We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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Software User Rights

1. With respect to KARL STORZ products containing software components, the Customer has a non-exclusive License, which is limited by the meaning of the following terms and conditions, to use the programmed logic, computer programs and/or software supplied by KARL STORZ and incorporated in KARL STORZ products ("Software").
2. This License applies for such products sold to the Customer only to the form delivered to Customer and for the sole purpose of operating such products in accordance with written instructions by KARL STORZ. Use of the Software in connection with other products bought by the Customer from KARL STORZ or from third parties is prohibited.
3. A transfer of this License by the Customer is permissible only if the Customer provides the buyer's name to KARL STORZ, and said buyer agrees to comply with the terms and conditions regarding the usage of the software as set forth in this License. In the case of a transfer, the Customer's License shall terminate.
4. All copyrights to the Software, as well as all modifications, enhancements, and upgrades thereto, will remain the property of KARL STORZ. In particular, the Customer may not duplicate, copy, reverse-engineer, decompile, or disassemble the Software or in any way modify the same. The Customer has no right to create derivatives of the software, and the Customer may not copy, decipher or re-create the source code of the Software. § 69g paragraph 2 of the Copyright remains unaffected by this.
[This refers to German Copyright Law]
5. Any modification or enhancement of the Software by the Customer in violation of this Agreement is considered a violation of this License; KARL STORZ immediately shall acquire all property rights and copyrights to such modifications or enhancements. The Customer hereby in advance transfers an exclusive, free, and unlimited license to use such modifications and enhancements to KARL STORZ.
6. KARL STORZ does not recommend use of the Software in connection with other software not supplied by KARL STORZ for the respective products. KARL STORZ is not liable for software malfunctions resulting from the use of software not supplied or recommended by KARL STORZ in connection with products supplied by KARL STORZ.
7. In the event of a failure of the Customer, or its agents, employees or representatives, to comply with any terms and conditions of the License herein granted, the License will, without termination or any further action by KARL STORZ or any other party, immediately terminate.

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Optional Accessories

Remote touch screen (15")	200903 02
Color printer kit (an isolation transformer 9370 TJ must be used with this printer).	
Dual pedal footswitch	200114 30
Dual pedal footswitch cable adaptor (required with footswitch)	200401 73
Wired lapel microphone	
CD discs, box of 10	202040 80
DVD discs (+RW), box of 5	202040 81
DVD discs (+R), box of 5	202040 82
External keyboard	202001 30U
External mouse	200902 33

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Appendix

Electromagnetic Compatibility (EMC) Information for AIDA DVD-M model 202045 20-1/140

WARNING: Medical electrical equipment needs special precautions regarding Electromagnetic Compatibility (EMC). Observe the EMC instructions in this appendix, as well as the instruction manual, during installation and commissioning.

The AIDA DVD-M model 202045 20-1/140 corresponds to EN/IEC 60601-1-2 :2001 [CISPR 11 Class A] and therefore meets the EMC requirements of the Medical Device Directive (MDD) 93/42/EEC.

These limits are designed to provide reasonable protection against the typical electromagnetic interference to be expected in a medical environment. The AIDA DVD-M model 202045 20-1/140 is a Group 1 unit (as per CISPR 11).

Group 1 contains all the 'equipment and systems which generate or use RF energy only for their internal functioning'.

NOTE: The tables and guidelines that are included in this appendix provide information to the customer or user that is essential in determining the suitability of the equipment or system for the electromagnetic environment of use, and in managing the electromagnetic environment of use to permit the equipment or system to perform its intended use without disturbing other equipment and systems or non-medical electrical equipment. If this equipment does cause harmful interference with other devices, the user is encouraged to try to correct the interference by one or more of the following measures:

- reorient or relocate the receiving device
- increase the separation between the equipment
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.

If you have any further questions, please contact your local representative or our service department.

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WARNING: The AIDA DVD-M model 202045 20-1/140 should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

WARNING: The AIDA DVD-M model 202045 20-1/140 is intended for use by healthcare professionals only. This is a CISPR 11 Class A medical system (RF emissions) and is suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes. In a domestic environment the AIDA DVD-M model 202045 20-1/140 may cause EMC interference, in which case it may be necessary to take adequate mitigation measures, such as re-orienting, relocating, or shielding the AIDA DVD-M model 202045 20-1/140 or filtering the connection to the public mains network.

WARNING: The use of portable and mobile RF equipment may have an impact on this and other pieces of medical equipment.

WARNING: The use of accessories, transducers and cables other than those specified in the KARL STORZ instruction manual, may result in increased emissions or decreased immunity of the AIDA DVD-M model 202045 20-1/140.

The cables listed in the following table (Table 200) have been shown to comply with the requirements of EN/IEC 60601-1-2. When using cables other than those specified here, it is the responsibility of the user to ensure that they comply with EN/IEC 60601-1-2.

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Table 200				
Cables which have been shown to comply with EN/IEC 60601-1-2:				
Cable type	Shielded	Length [m]	Ferrite	Used for
Y/C [S-Video]	Yes	1.8	No	Connection of S-Video signals
BNC to BNC	Yes	1.8	No	Connection of composite or SDI video signals
BNC to RGB Video	Yes	1.8	No	Connection of RGB signals
4 Pin Mini DIN	Yes	1.8	No	Connection of SCB signals
1395 Firewire	Yes	4.5	No	Connection of DV Signals
AC Power Cord	No	3	No	Connection of device to mains
Stereo Mini Phone 3.5 mm	Yes	1.8	No	Connection of AIDA DVD-M to CCU via accessory ports of CCU

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Table 201		
Guidance and manufacturer's declaration – electromagnetic emissions		
AIDA DVD-M model 202045 20-1/140 is intended for use in the electromagnetic environment specified below. The customer or user of the AIDA DVD-M model 202045 20-1/140 should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	AIDA DVD-M model 202045 20-1/140 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The AIDA DVD-M model 202045 20-1/140 is suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

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Table 202			
Guidance and manufacturer's declaration – electromagnetic immunity			
AIDA DVD-M model 202045 20-1/-140 is intended for use in the electromagnetic environment specified below. The customer or user of the AIDA DVD-M model 202045 20-1/-140 should ensure that it is used in such an environment.			
Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	Complies ± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Complies ± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment. If image distortion occurs it may be necessary to further filter the power quality by using a AC line filter.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Complies ± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T * (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	Complies <5% U_T * (>95% dip in U_T) for 0.5 cycle Complies 40% U_T (60% dip in U_T) for 5 cycles Complies 70% U_T (30% dip in U_T) for 25 cycles Complies <5% U_T (>95% dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment or system requires continued operation during power mains interruptions, it is recommended that the equipment or system be powered from an uninterruptible power supply or a battery. If image distortion occurs it may be necessary to further filter the AC input to the AIDA DVD-M model 202045 20-1/-140 by using an external AC line filter.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	Complies 3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
* Note: U_T is the a.c. mains voltage prior to application of the test level.			

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Table 204 Guidance and manufacturer's declaration – electromagnetic immunity – for equipment and systems that are not life-supporting			
AIDA DVD-M model 202045 20-1/-140 is intended for use in the electromagnetic environment specified below. The customer or user of the AIDA DVD-M model 202045 20-1/-140 should ensure that it is used in such an environment.			
Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 V_{me} 150 kHz to 80 MHz 3 V/m 80 MHz to 2.5 GHz	3 V_{me} 3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the AIDA DVD-M model 202045 20-1/-140, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = [3.5/3]\sqrt{P}$ $d = [3.5/3]\sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = [7/3]\sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ Where P is the maximum output power rating of the transmitter in watts [W] according to the transmitter manufacturer and d is the recommended separation in meters [m]. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b. Interference may occur in the vicinity of equipment marked with the following symbol: 
Note 1: At 80 MHz and 800 MHz, the higher frequency range applies Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection by structures, objects and people.			
^a Field strengths from fixed transmitters, such as base stations for radio [cellular/cordless] telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the unit is used exceeds the applicable RF compliance level above, the unit should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the unit. ^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

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Table 206

**Recommended separation distances between portable and mobile RF communications equipment and the
AIDA DVD-M model 202045 20-1/-140**

The AIDA DVD-M model 202045 20-1/-140 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the unit can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the unit as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter [W]	Separation distance d [m] according to frequency of transmitter		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{3} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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WARRANTY

For a period of **two** years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship.

In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

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KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen/Германия

To be filled out by instrument owner:	
Return address/company stamp:	
Field of application:	
Type of instrument: Serial no.:	
Purchase date:	
Signature/Date:	

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KARL STORZ Endoscopy China Ltd.
Hong Kong Representative Office
Unit 1601, Chinachem Exchange Square
1 Hoi Wan Street, Quarry Bay, Hong Kong,
People's Republic of China
Phone: +8 52 28 65 2411
Fax: +8 52 28 65 4114
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Beijing Representative Office
Room 610, China Life Tower
No. 6, Chaowai Street
Beijing, 100020, People's Republic of China
Phone: +86 10 8525 3725
Fax: +86 10 8525 3728
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Unit 3901-3904, Tower 1 Grand Gateway
No. 1 Hong Qiao Road
Shanghai, 200030, People's Republic of China
Phone: +86 21 6113-1188
Fax: +86 21 6113-1199
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Chengdu Representative Office
F-5, 24/F., Chuanxing Mansion
No. 18 Renming Road South
Chengdu, Sichuan, 610016, People's Rep. of China
Phone: +86 28 8620-0175
Fax: +86 28 8620-0177
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Guangzhou Representative Office
Room 1119-20, Dongshan Plaza
69 Xianlie Road Middle, Dongshan District,
Guangzhou, Guangdong, 510095,
People's Rep. of China
Phone: +86 20 8732-1281
Fax: +86 20 8732-1286
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy Japan K. K.
Bois Hongo Building 6F1, 3-42-5 Hongo
Bunkyo-ku, Tokyo 113-0033, Japan
Phone: +81 3 5802-3966
Fax: +81 3 5802-3988
E-Mail: info@karlstorz.co.jp

KARL STORZ GmbH & Co. KG

Mittelstraße 8
78532 Tuttlingen

Postfach 230
78503 Tuttlingen
Germany

Telefon: +49 (0)7461 708-0
Telefax: +49 (0)7461 708-105
E-Mail: or1@karlstorz.de
Web: www.karlstorz.com