Table of Contents

WARRANTY AND SERVICE	6
UNPACKING, ASSEMBLY AND INITIAL SYSTEM CHECK	7
INSPECTION SCHEDULE	8
CHAPTER 1. INTRODUCTION	9
1.1. Product Description	9
1.2. General Warnings and Cautions	9
1.3. Intended Use	9
1.4. Symbol Definitions	10
1.5. System Indicators	12
A. Front Panel	12
B. Rear Panel	14
1.6. Other Indicators (Audible Tones)	14
CHAPTER 2. OPERATING PRECAUTIONS, CHECKS & WARNINGS	15
2.1. Storage and Handling	15
CHAPTER 3. SET-UP PROCEDURES	16
3.1. Pump Set-up	16
3.2. Tubing Set-up	17
A. Installing the One Day Set Irrigation Tube	17
B. Installing the Intermediary Irrigation Tube	19
C. Getting Started	20
D. End of Operation	21
E. Set-up for Next Operation	22
F. End of Operating Day	23

CHAPTER 4. PUMP SYSTEM FUNCTIONS	24
4.1. Run/Stop	24
4.2. Preset Pressure	24
4.3. Filling the Chamber	24
4.4. Recommended Initial Settings	25
4.5. Pressure Alarm	25
4.6. Pressure Cycle Mode	26
CHAPTER 5. MAINTENANCE: CLEANING & DECONTAMINATION	27
CHAPTER 6. TROUBLESHOOTING	28
CHAPTER 7. PRODUCT SPECIFICATIONS	30

Warranty and Service

DePuy Mitek products are guaranteed to be free from defects in material and workmanship for the warranty period of a particular product, beginning from date of invoice.

Contact your DePuy Mitek representative for specific product warranty information.

DePuy Mitek offers an expedited service replacement program for its products to minimize downtime in your operating room. For a return authorization number or for additional information on this program, call DePuy Mitek Customer Service at +1 (800) 382-4682 and choose Option 2 in the United States or contact your DePuy Mitek representative.

Unpacking, Assembly and Initial System Check

Unpacking:

Verify that all items on packing list match the products you have received (verify serial numbers and references). Damage should be reported immediately to your DePuy Mitek Sales Representative or contact DePuy Mitek Customer Service in the USA at +1-800-382-4682, select Option 2. Save all packaging material, as they will be required to ship products in the event of a product return.

Once box is open, remove, read, and follow the *Instructions for Use*. To remove the FMS SOLO console, place hands on both sides of console, until you can securely grasp the underside. Carefully remove the console from the box and place it on a level and sturdy surface.

Assembly:

- Connect the power cord to the power connection (14).
- Connect the foot pedal to the foot pedal connection (16) on the back of the pump.

Initial System Check:

- Switch the I/O switch (14) on "I".
- Turn the device ON by pressing the POWER (4) pad.
- Press on all pads. An audible tone should be heard for each pad pressed, except for Pressure Cycle, (Pad 8) (Pressure Cycle is inactive until the pump is primed.)

Note: numbers in (), refer to diagram in section 1.5.

Inspection Schedule

Weekly inspection:

DePuy Mitek recommends a weekly inspection of all equipment. These inspections can be performed on a less frequent basis depending on the duration and frequency of use.

- Perform a visual inspection of the device. If damage is noted, contact your DePuy Mitek Sales Representative or call DePuy Mitek Customer Service in the USA at +1-800-382-4682. Select Option 2 to speak to a biomed.
- Inspect that the transparent safety cover (1) and black hinges are in good working order.
- Inspect power cord and all cables for wear or cuts.
- Ensure that the foot pedal and shaver interface connector pins are not missing or bent.
- Plug the power cord to the power connection.
- Switch the I/O switch (14) on "I".
- Turn the device ON by pressing the POWER (4) pad.

Press on all pads. An audible tone should be heard for each pad pressed, except for Pressure Cycle, (Pad 8) Pressure Cycle is inactive until the pump is primed.

Annual inspection:

DePuy Mitek recommends that a bio-medical engineer or authorized service personnel annually check all equipment to assess its functionality and technical safety.

Chapter 1. Introduction

1.1. Product Description

Irrigation system.

The FMS SOLO is an advanced, lower cost irrigation system that uses patented technology to improve visualization and distension while controlling pressure during arthroscopic surgery.

Flow compensation.

The FMS SOLO can recognize when a shaver is in rotation and subsequently improve by activating the "Flow compensator" cycle. Please contact your DePuy Mitek representative for FMS Shaver Interface products due to the variety of products offered.

Pressure cycle red pedal.

One of the most critical aspects of arthroscopy is visualization. The field of view is often obstructed by blood and debris. To alleviate this problem the FMS SOLO has a pressure cycle feature. By stepping on the red foot pedal you will activate a timed cycle of increased pressure to control bleeding. Once the cycle is completed, the FMS SOLO will return to its original configuration.

1.2. General Warnings and Cautions

It is recommended that hospital personnel read this manual before operating or cleaning this product and its accessories. Failure to follow these instructions may result in potential injury and damage or malfunction of the equipment.

The manufacturer and licensed seller of this device do not accept any liability for direct or consequential damage or injury caused by improper use of disposables other than FMS products. Any alterations to this device, repair from an unlicensed service center, or use of non-FMS disposables, may void CE marking, FMS warranty and product liability coverage.

The tube sets contain DEHP, a phthalate chemical known to cause birth defects and other reproductive harm, including in children and in pregnant or nursing women.

Warning:

The FMS SOLO is a Class I Type B device according to the Electrical Safety Standard EN 60601-1 for medical devices. Ensure that you are using the correct interface cable when connecting to another shaver. For more details, refer to Interface Cable Instructions for Use. Contact your DePuy Mitek Sales Representative to review Shaver Interface Cable interface offerings.

The system may be affected by electromagnetic interferences from other instruments. Verify that all other instruments and appliances (associated or not with the system) comply with the standard EN 60601-1-2 (EMC).

If the system continues to be affected, isolate it from the offending instrument and connect to a different mains outlet.

1.3. Indication for Use

The FMS SOLO Arthroscopic Fluid Management System is indicated for use in a surgical setting by personnel trained in arthroscopy.

1.4. Symbol Definitions

Front Panel	Back Panel
 Power pad Turns the power ON & OFF if the main power switch on the back of the pump is on (the green standby light is on).	 On
 Alarm The red indicator will light up if a safety parameter is violated.	 Off
 Pressure cycle Alleviates bleeding during surgery, by increasing base pressure for two minutes.	 Equipotential symbol (ground)
 One-way foot pedal board Used to activate or cancel a pressure cycle.	 No protection from liquid
 Increase	 Type B Class 1 Conforms to standard EN 60601-1.
 Decrease	 Caution See instructions for use.
 Refer to instruction manual/ booklet	
 Refer to instruction manual/ booklet	

Explosion hazard

Main fuse

Replace fuses as marked


Per Directive 2002/96/EC,
product must be collected
separately. Do not dispose of
as unsorted municipal waste.
Contact local distributor for
disposal information.

Do not wet

1.5. System Indicators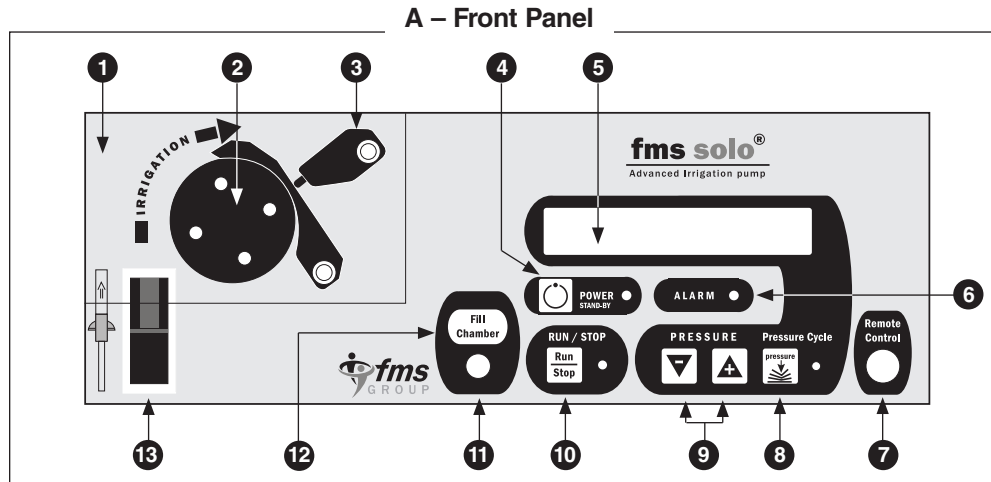


- 1 Transparent safety covers.**
This cover **must** be closed for the roller pump to rotate.
- 2 IRRIGATION roller pump.**
Provides fluid inflow.
- 3 Tension rocker arm.**
Holds the tubing on the roller pump head.
- 4 POWER pad.**
Turns the power **ON & OFF** if the main power switch in the back of the pump is on (the green “standby” light is on).
- 5 PRESSURE display.**
Displays both the preset pressure when the base pressure is being set, and the dynamic pressure (1 sec) following the release of the pressure adjustment pads.
- 6 ALARM.**
The red indicator will light up if a safety parameter is violated.
- 7 Remote Control connector.**
Connects the remote control to the FMS SOLO.

- 8 Pressure cycle.**
Alleviates bleeding by increasing base pressure for a 2 minute cycle.
- 9 PRESSURE adjustment pads.**
Press the arrows to increase or decrease the pressure in increments of 5. The initial setting is 50.
- 10 Pump Run/Stop pad.**
Turns the pump ON and OFF. If the pump is activated, <PUMP> appears on the pressure display. Green LED blinks when pump is on stop.
- 11 Built-in Pressure transducer connection.**
To connect the One Day Set Irrigation Tube and measure the pressure in the pressure chamber.
- 12 Fill Chamber pad.**
Manually fills the chamber and turns off the LOW PRESSURE alarm.
- 13 Patented auto-locking device.**
Allows the disposable tubing to be properly positioned and locked in place around the roller pump.

B – Rear Panel

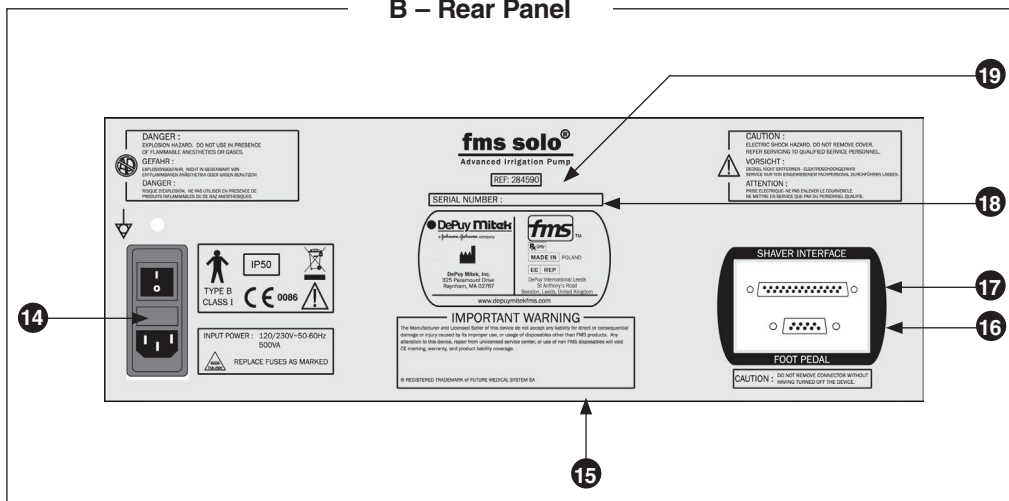14 I/O switch, fuses and power connection.

15 Identification & specification.

16 Foot pedal connection.
9-pin connector to plug foot pedal.

17 Shaver interface connection.
25-pin hand and foot control interface cables connector. Contact your DePuy Mitek Sales Representative for Shaver Interface Cable ordering information due to the variety of products available.

18 Serial number.

- The first two digits represent the year of manufacturing.
- The second set of digits represents the week of manufacturing and serial number.

19 Reference number.

1.6. Other Indicators (Audible Tones).

- An audible tone sounds when the **POWER** pad is pressed and the pump turns on.
- An audible tone sounds when the **PRESSURE** mode is activated (see pressure cycle section to activate the cycle).
- Two audible tones sound when the **PRESSURE** mode is deactivated (see chapter 4 to deactivate the cycle).

Chapter 2. Operating Precautions, Checks and Warnings

Warnings:

Factory Settings: Factory settings are based on experience; they should only be used as guidelines. The surgeon is responsible for settings depending on the surgical procedure performed.

Electrical safety testing should be performed by a biomedical engineer or other qualified person.

Risk of electrical shock: Do not remove the cover. Refer servicing to your DePuy Mitek Representative or DePuy Mitek Customer Service in the USA at +1 (800) 382-4682, Option 2.

Do not connect the device to a power source that is not properly earthed (grounded).

Disconnect the device from the main power source when cleaning, servicing, or inspecting.

Inspect all equipment and cables periodically for wear. Contact your DePuy Mitek Representative or DePuy Mitek Customer Service in the USA at +1 (800) 382-4682, Option 2 if damage is noted.

Avoid fluid contact with the FMS SOLO, and its electrical connectors.

Do not use flammable agents when cleaning and disinfecting the FMS SOLO.

To avoid risk of fire, replace fuses with same type and rating.

2.1. Storage and Handling:

Storage temperature: Do not expose the FMS SOLO to temperatures above 50°C (122°F) or below -10°C (14°F).

Operating Conditions: Do not operate the pump in conditions above 40°C (104°F) or below 10°C (50°F).


Damage: Do not use a FMS SOLO that has been dropped or shows signs of damage.

Disposal: Do not incinerate.


Contact your DePuy Mitek Representative or DePuy Mitek Customer Service in the USA at +1 (800) 382-4682, Option 2 to arrange for safe disposal.

Chapter 3. Set-up Procedures

3.1. Pump Set-up.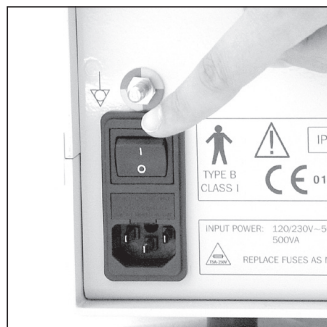

Power on back of the pump on "1".

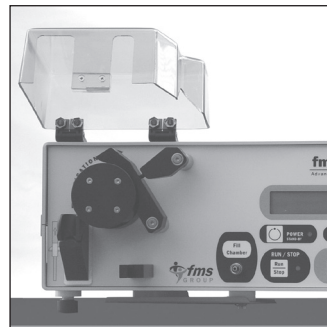Open the cover.

Press Power pad.
The LCD will light up.

3.2. Tubing Set-up.

WARNING: This product is for single-use only. It has not been designed to be re-used/re-sterilized. Reprocessing may lead to changes in material characteristics such as deformation and material degradation which may impact the strength of the device and compromise device performance. Reprocessing of single use devices can also cause cross-contamination leading to patient infection. These risks may potentially affect patient safety.

Sterility is preserved only if the tube changing procedures have been performed precisely according to the set-up instructions below and in Section 3.2.E.



This product contains DEHP, a phthalate chemical known to cause birth defects and other reproductive

harm, including in children and in pregnant or nursing women.


This tube connects the saline solution bags to the irrigation tubing. Solution goes from the spikes (A) around the roller pump, through the pressure chamber, where the pressure is read through a filter (B) by a built-in pressure transducer.

A. Installing the One Day Set Irrigation Tube Ref. 284504 or 284503


Contact your DePuy Mitek Representative for ordering information since certain tubes are not offered in all countries.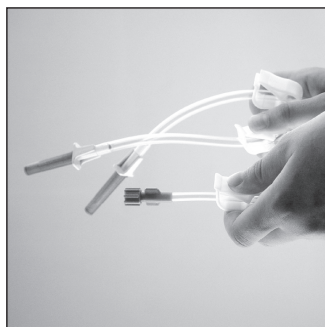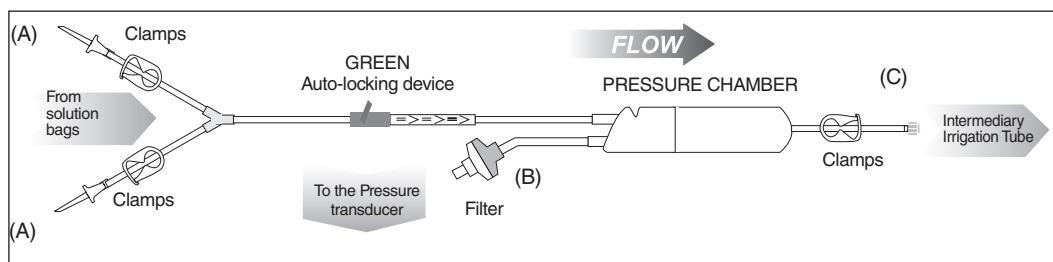

Close all three clamps and connect to fluid bags.

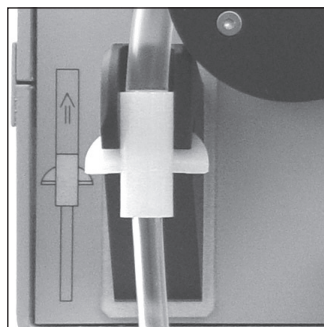
Slide the green part of the tube into the auto-locking device on the pump.

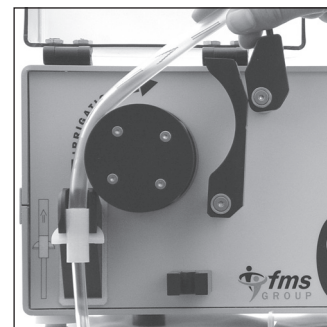
Place and center the tube around the roller pump.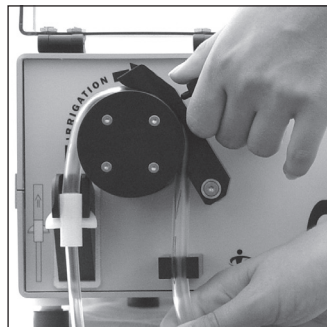


Pull on the tube and close the tension rocker arm against the tube.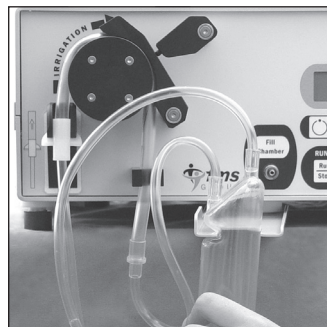

Position the pressure chamber in the holder. Connect the transducer to the pump by firmly screwing it in clockwise.

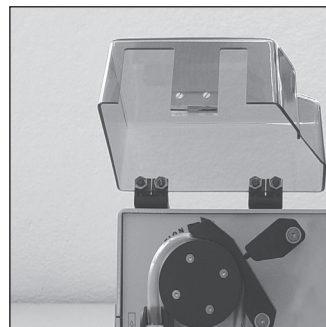Close the cover.

Green LED blinks. Press Run/Stop. The green LED blinks initially, then remains on.

Press the Fill chamber pad several times until the chamber is 1/3 full. The green LED blinks until pad is pressed.


Warning:

It is mandatory that the pressure chamber remains vertical during the procedure and that the pump be positioned at the same level as the patient.

Over filling the chamber may result in the filter (shown in 3.2.A) becoming wet prior to the pressure transducer and therefore interfere with accurate pressure measurement. Should over filling of the chamber occur, see Chapter 6 for Troubleshooting.


B. Installing the Intermediary Irrigation Tube Ref. 281103, 281142, or 281103*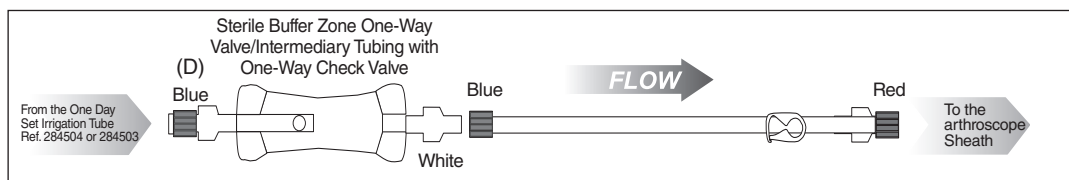


This tube connects the irrigation tube to the sheath in a sterile manner.


* Ref. 281103 does not include the Sterile Buffer Zone One-Way Valve/Intermediary Tubing with One-Way Check Valve. Ref. 281103 is intended for single use of the One Day Set Irrigation tube.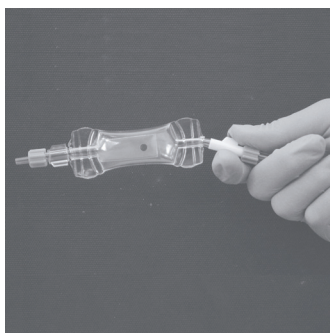

After checking that the luer-lock is properly tightened, the sterile nurse takes the irrigation tube and passes the blue end (D) to a non-sterile person.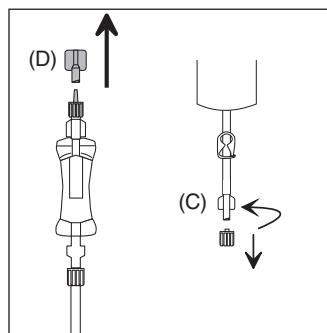

The non-sterile person removes the blue caps from the irrigation tube (D) and the One Day Set Irrigation Tube (C)...

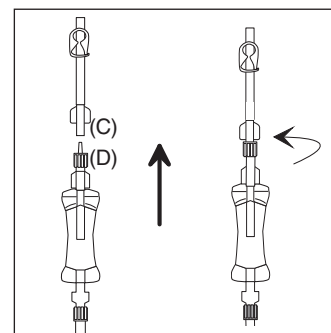
... and **immediately** connects the One Day Set Irrigation Tube (C) luer-lock, to the irrigation tube (D) luer-lock.


The pump must be in the Run mode (10), with the light on. Prime the irrigation tube by opening the clamp under the pressure chamber. When primed, close the clamp or the stop-cock on the arthroscope (in the sterile zone).


C. Getting Started.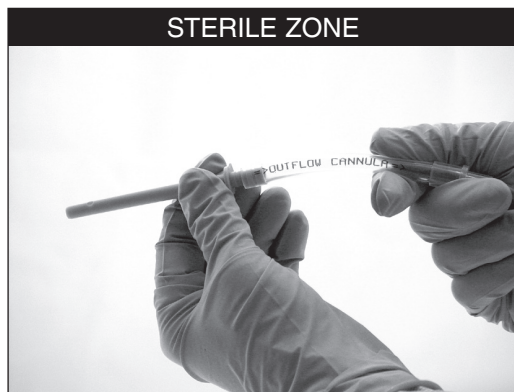


Insert the cannula tube onto the cannula.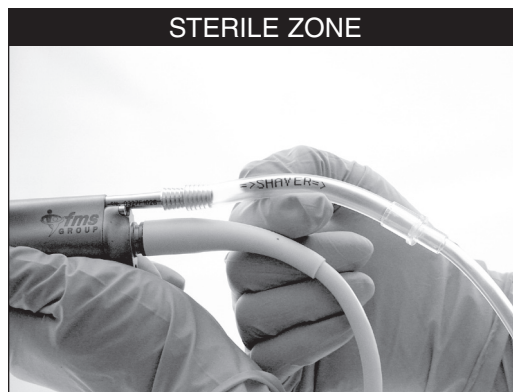

Insert the shaver tube onto the shaver suction port.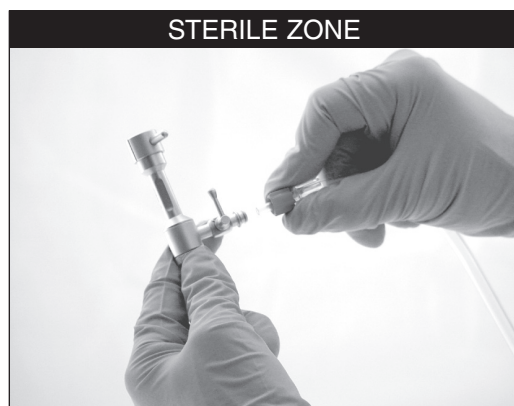


Connect the luer-lock to the arthroscope. Open the stop-cock on the scope.

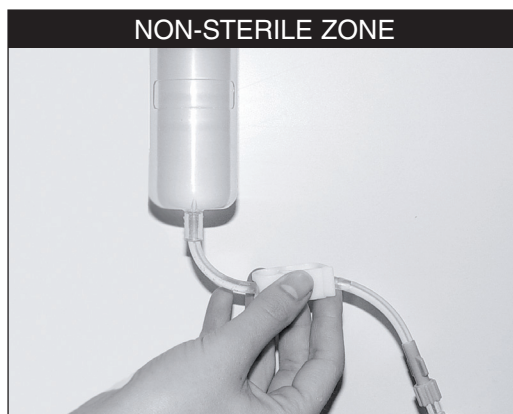Open the clamp under the chamber.


D. End of the Operation.

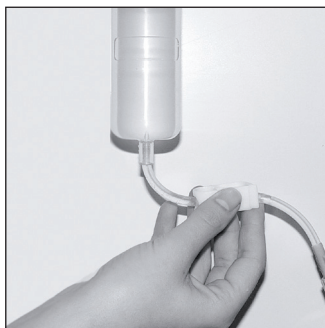Close the clamp under the pressure chamber.

Press Run/Stop pad (or open transparent safety cover).

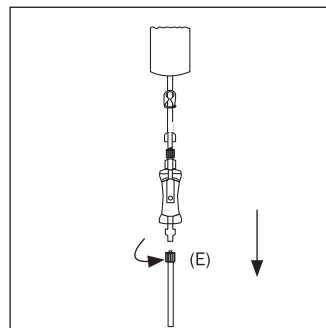Disconnect the blue luer-lock (E) of the Intermediary Irrigation tube.


Note: Ref. 284503 & 284504 “One Day Set” can be re-used for multiple procedures during the same surgical day. **Warning:** Sterility is preserved only if proper set-up and tube changing procedures have been followed.


Warning: The Sterile Buffer Zone One-Way Valve/Intermediary Tubing with One-Way Check Valve must remain attached to the One Day Set Irrigation Tube which ensures sterility between surgeries.

E. Set-up for Next Operation.


The One Day Set Irrigation Tube can be left in place at the end of the previous operation with the Sterile Buffer Zone One-Way Valve/Intermediary Tubing with One-Way Check Valve attached.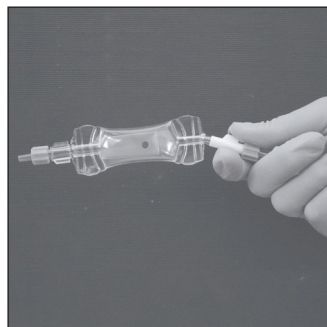


After checking that the luer-lock is tightened, the sterile nurse takes the irrigation tube and passes the blue end (D) to a non-sterile person.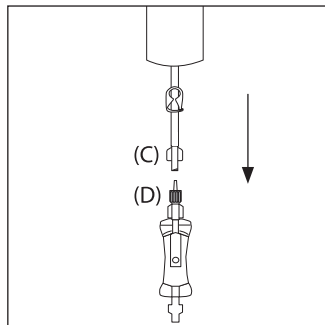

The non-sterile person removes the old sterile buffer zone by disconnecting the luer-locks of the One Day Set Irrigation Tube (C) and the Intermediary irrigation tube...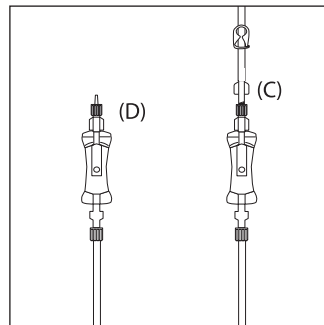


... and **immediately** connects the blue luer-lock (D) of the new irrigation tube to the blue luer-lock (C) of the One Day Set Irrigation Tube.

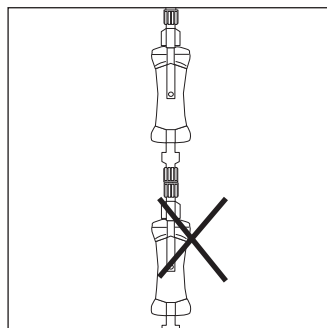
Warning:
Never attach one Sterile Buffer Zone One-Way Valve/Intermediary Tubing with One-Way Check Valve to another.

F. End of Operating Day.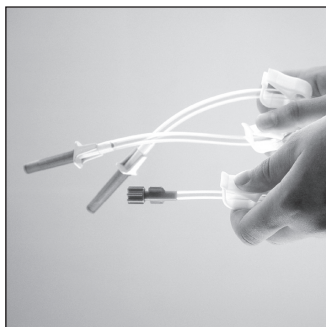


Close all white clamps.

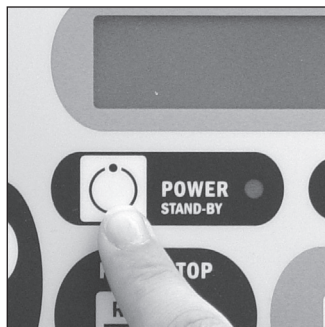
Turn off the pump by pressing the POWER pad. Throw away all of the tubing.

Chapter 4. Pump System Functions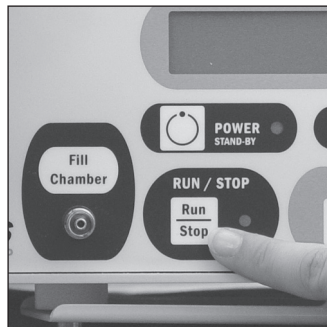

4.1. Run/Stop

Press the Run/Stop pad to start all pump system functions.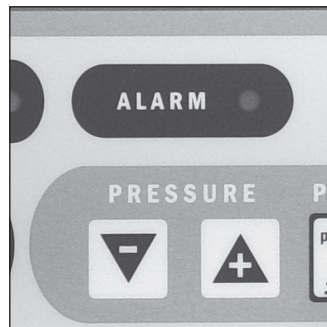


4.2. Preset Pressure

The pressure is displayed in brackets (eg. <50>) on the LCD (5) for 2 seconds when one of the pressure pads is pressed, then the actual pressure is displayed (pressure settings are in levels).

Level 50, equals 5 feet of gravity pressure or the equivalent of 112.5 mmHg ($\pm 10\%$).

The pump will maintain the pre-set level. When operating, use the “+” and “-” pads (9) to increase and decrease the level. The factory preset level is 50.

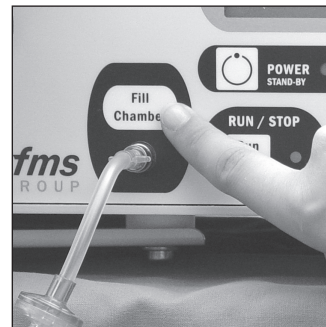4.3. Filling the Chamber

Press the Fill Chamber pad to fill the pressure chamber initially, and throughout the surgery as needed. The chamber should never be over 1/3 full.

Note: numbers in (), refer to diagram on page 12.

4.4. Recommended Initial Settings. *

Joint	Pressure level with Tourniquet	Pressure level without Tourniquet
Shoulder Joint		60
Acromioplasty		60
Knee Joint Hemarthrosis	30 50–60	65 50–60
Wrist	30	65
Elbow, Ankle	40	65
Hip		65

Note: If bleeding occurs, the surgeon may choose to activate the Lavage mode or increase pressure.

*  **WARNING:** These initial settings are based on experience and should only be used as guidelines. The surgeon is responsible for setting depending on the surgical procedure performed.

4.5. Pressure Alarm

**ALARM**

High pressure alarm.

If the high pressure safety level of 140 is exceeded, the pump stops and the red ALARM (6) LED lights up. The operator should lower the pressure to let the pump automatically restart. If the pump does not restart when the pressure drops below 140, restart the pump manually by turning the pump ON and OFF again with the POWER pad.

Low pressure alarm.

If the pressure falls below the low pressure safety level of 10, the red ALARM LED lights up, <Pres L> flashes on the LCD display, and the pump stops. Check that the luer-lock and the filter near the pressure transducer are in proper working order. If both are satisfactory press Fill Chamber to stop the alarm and Run/Stop pad to restart the unit.

Warning:

If water has accidentally entered the tubing leading to the pressure transducer, change the One Day Set Irrigation Tube and restart the system.

Note: numbers in (), refer to diagram on page 12.

4.6. Pressure Cycle Mode.

This function increases the pressure by 60% for a two minute cycle. The pressure then returns automatically to its original setting. The pressure cycle mode is activated when bleeding occurs. The pressure cycle can be canceled by pressing the Pressure Cycle mode again.


The pressure cycle mode is activated either by pressing the "Pressure" pad...

... or using the remote control Ref. 288051, which is connected on the front panel of the pump...

The remote control is not autoclavable and not soakable. A sterile disposable remote control cover must be used Ref. 281087.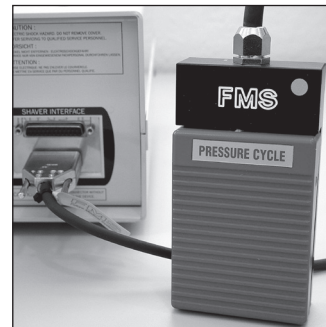

... or by pressing on the red foot pedal Ref. 284171, which is connected on the rear panel of the pump (see on page 14, n° 16).

Chapter 5. Maintenance: Cleaning & Decontamination

Cleaning the FMS SOLO:

At the end of the operating day, switch the FMS SOLO off, unplug the power cord and discard all tubing. Wipe off the instrument's sides, front panel and covers with a soft cloth that has been slightly moistened with a neutral-pH detergent. Wipe again using distilled water to rinse.

Warning:

DO NOT IMMERSE. DO NOT AUTOCLAVE. Do not use flammable agents when cleaning and disinfecting the FMS SOLO.

Foot Pedal: Cleaning and Decontamination

After each operation, clean the foot pedal and the power cord with a neutral-pH detergent. The foot pedal board can be immersed, except the electrical connections, which must remain dry.

Warning:

DO NOT IMMERSE ELECTRICAL CONNECTOR OF FOOT PEDAL. DO NOT AUTOCLAVE. Do not use flammable agents when cleaning and disinfecting the Foot pedal.

Do not twist the cable when storing.

Do not unplug instrument by pulling on cable.

Chapter 6. Troubleshooting

Problems	Potential Causes	Possible Solutions
The pump stops. Red ALARM LED lights up.	The HIGH PRESSURE safety has activated.	Lower the joint pressure. Press the Run/Stop pad to restart the pump.
	The LOW PRESSURE safety has activated: "Pres. L" blinks.	Verify that the luer-lock is tightly closed. Press the Fill Chamber to stop the alarm, and press the Run/Stop pad to restart the pump.
	The hydrophobic filter is wet.	Replace the One Day Set Irrigation Tube. If problem persists, contact DePuy Mitek Customer Service.
	There is no more irrigation fluid.	Spike the bag and press the Fill Chamber pad.
The roller pump heads do not rotate.	The power is on "O" or is on Stop.	Check that the main power switch on the back of the pump is on the "I" position, and that the Run/Stop pad green light is on.
There is no irrigation.	The stop-cock on the sheath or a clamp is closed, stopping outflow.	Check the inflow line from the bags to the joint. Open any closed clamps or stop-cocks.
The irrigation pump starts spinning excessively, emitting a loud noise.	There is no more irrigation fluid.	Replace the saline solution bags and/or check the clamps under the bags.
		Press Fill Chamber if chamber is lower than 1/3.

Problems	Potential Causes	Possible Solutions
The irrigation pump goes on and off erratically causing abnormal pressure fluctuations.	Water may have entered the pressure sensing line.	Change the One Day Set Irrigation tube if water has entered the pressure sensing line. If problem persists, contact DePuy Mitek Customer Service.
There is too much water in the pressure chamber.	An air leak in the line may have developed.	Check that the luer-lock on the irrigation and pressure sensing tube are closed. If necessary, change the irrigation tubing and start again.
There is insufficient pressure in the system.	An irrigation problem exists.	Check the stop-cock on the arthroscope, and the clamps under the saline bags and under the pressure chamber.

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